**GEBRAUCHSANLEITUNG
DIRECTIONS FOR USE
NOTICE D'UTILISATION
GEBRUIKSHANDLEIDING
MANUALE PER L'USO
MANUAL DO UTILIZADOR
MANUAL DE INSTRUCCIONES**

POWERTEC F35

**SUNRISE
MEDICAL**

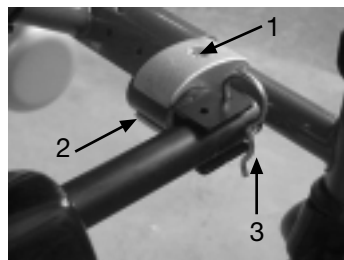
Fig. 5

Fig. 6

Fig. 7

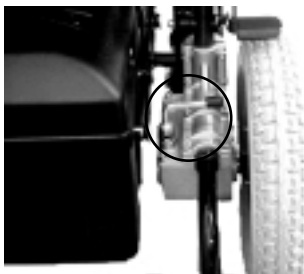
Fig. 8

Fig. 9

Fig. 10

Fig. 11


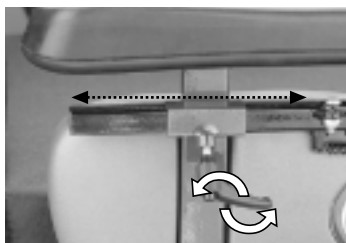
Fig. 12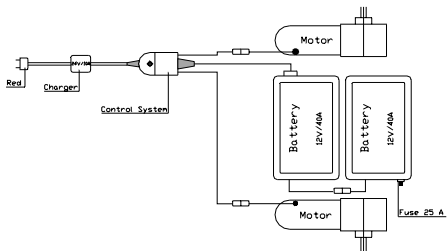

Verkabelungsschema
 Wiring Diagram
 Schéma du câblage
 Bekabelingsschema
 Schema del cablaggio
 Esquema da cablagem
 Esquema del cableado

EINLEITUNG

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INTRODUCTION

Thank you for choosing a SUNRISE MEDICAL product. In view of the broad range of components and accessories available for all SUNRISE MEDICAL wheelchairs it is in your interest to consult a health professional or authorised SUNRISE MEDICAL distributor to be sure that your wheelchair is properly adjusted for your safety and comfort. It is advisable to read this instruction manual, especially the safety advice and the sections regarding adjustment, before using the wheelchair.

The F35 wheelchairs are designed for persons with a physical handicap, to be used only for transport. These wheelchairs are suitable for use both indoors and outdoors (Class B product).

Accessories:

- Kerb climber
- Elevating legrest
- Chin control
- User and companion control
- Companion control

If you have any questions regarding use, maintenance, or safety of your SUNRISE MEDICAL wheelchair, please consult one of our authorized distributors.

Note: The user of this product should be examined by a health professional so that it can be verified that the appropriate model has been prescribed with the appropriate accessories. The user should be advised by a health inspector so that he may be sure his own limitations and physical condition do not hinder the safe use of a wheelchair in the course of daily life. The maximum user weight is limited at 120kg and the maximum incline rated at 10° (18%).

Clean the frame regularly with a non abrasive cleaner, subsequently applying polish with nonabrasive wax. You can clean off small stains with a sponge moistened in gentle detergent and warm water. Rinse with water. Do not put upholstery in a dryer.

Sunrise Medical LTD.
Sunrise Business Park
High Street, Wollaston
West Midlands DY8 4PS
England
Phone: 1384 44 66 88
Fax: 1384 44 66 99

Dealer stamp

Sunrise Medical is ISO 9001 certified, which ensures quality at all stages of the development, and production of this wheelchair.

This product is manufactured to comply with the radio interference requirements of EEC directive 89/336/EEC Medical Device Directive 93/42/EEC.

INDEX

A.SAFETY	14	D.USE	16
B.CHARACTERISTICS	15	E.ADJUSTMENTS	19
C.ASSEMBLY	15	1. Footrest	
1. Footrest		2. Armrest	
2. Armrest		3. Control position	
3. Anti- tip wheels		4. Programming the control	
4. Kerb climber		F. CONTROLS AND MAINTENANCE ...	20
5. Batteries		G.SPECIFICATIONS	20
6. Wheels		H.GUARANTEED	20
7. Seat belt		I. SERVICE HISTORY	21

A. SAFETY

If you follow the instructions given in this manual, you will enjoy many trouble-free years of use. This vehicle is not designed for use on the road except for crossing from one kerb to another.

1. Preliminary controls: You should not try to manoeuvre at full speed. If it is necessary to slow down first by using the joystick or the speed control knob. This is very important if you are going down a hill. Lack of attention to this point may result in your overturning the wheelchair. Always check that the wheelchair is switched off before trying to get in or out. Always check that you can operate all the controls from a comfortable position. It is essential to give attention to posture in order to assure comfort and well-being. Always check that the freewheel device is correctly coupled before setting the chair in motion. Always check that you can be seen clearly, especially if you are using the wheelchair in conditions of low visibility.

2. Emergency braking: If power is cut by means of the ON/OFF button, braking will be immediate and total. This method of stopping is not recommended except for emergencies, since the stop will be extremely sudden.

3. Kerbs

- Give due attention to other road users. Remember that the last thing a car or truck driver expects to see is a wheelchair descending onto the road from a kerb. If you are at all doubtful, do not cross the road until you have checked that there is nothing coming.
- Always cross the road as rapidly as possible, since other vehicles may appear.
- Do not attempt to climb, or to descend from, a kerb of more than 10 cm. height.
- Do not attempt to ascend high kerbs, steep slopes or cambers due to the risk of falling out of the chair or tipping over.

- Do not attempt to ascend any kerb in the vicinity of a drain cover, uneven surfaces, or gravel covering.
- Do not attempt to descend a kerb of more than 5 cm in forward position.
- Do not attempt to climb or descend any kerb at an oblique angle. Approach only at an angle of 90° with approx. 0,5m run-up.

4. Tyres: The tyres of a wheelchair suffer wear depending on how much the chair is used. Check them regularly, especially for inflation pressure, according to the service instructions in this manual. Caution: NEVER INFLATE the tyres with the air hose of a service station.

5. Freewheel device: Only use this function to push the chair manually from one spot to another. Keep in mind that while the freewheel system is in operation, the chair is without a braking system.

6. Transportation in vehicles: This wheelchair has been crash tested in its standard configuration. However, due to safety risks Sunrise Medical do not recommend the transportation of people in their wheelchairs.

Should a person need to be transported in wheelchair on transport systems, the following should be adhered to:

1. The wheelchair must be positioned forward facing.
2. For tie down points on the wheelchair, please look for the tie-down symbols on the wheelchair.
3. A head restraint must be fitted and suitably positioned at all times during the transportation.
4. If possible, remove seat cushion to create a lower centre of gravity.
5. The chair is crash-tested, and has satisfied the Dynamic test requirements of ISO 7176-19.
6. The wheelchair must be secured by a 4-part Tie Down Restraint system, conforming to ISO 10542 part 2.
7. The occupant must be restrained independently of the wheelchair by a lap and diagonal safety belt, conforming to ISO 10542 part 3. Sunrise Medical postural lap belts are for postural support only, and not suitable as restraints during transportation.
8. Any detachable accessories or components of the wheelchair must be removed and stored securely in the vehicle luggage compartment during transportation.
9. The Tie down restraints should be fitted to the main frame of the wheelchair as indicated by the karabiner stickers, and in the User Manual, and not to any other part of the chair.
10. The Tie Down restraints should be attached as close as possible at an angle of 45°, and tightened securely in accordance with the restraint manufacturers' instructions.
11. The wheelchair parking brakes must be firmly applied.

7. Mobile telephones and radio telephones: Neither radio telephones nor mobile phones should be used while the wheelchair is being driven, since they may give rise to excessively strong electromagnetic fields that could interfere with the electronic systems of the wheelchair. If you have to use such a phone, stop and switch off the

power. Keep away from aerials, antennas and high voltage electricity pylons. Take into account that the wheelchair itself can disturb electromagnetic fields such as emitted by alarm systems of shops.

8. Contact with water, snow or ice may affect the performance of your wheelchair. Never use your wheelchair in a pool or other body of water. If your chair becomes wet, or if you use water to clean it, dry the chair thoroughly as soon as you can.

9. Hot surfaces: Not only motors can be hot during the operation of the chair, but also the upholstery material and armrests when standing in the sun.

10. Lap belts: Lap belts are fitted as positional aids, and are not suitable as transportation restraints.

Position belt is situated in the middle of the stomach. The positioning belt should be adjusted so that a hand can be placed between the user and the belt.

Maintenance: Check lap belt, and security components, at regular intervals for any sign of frays, or damage.

Replace if necessary.

WARNINGS:

- The rider plus item should never exceed a total weight of 120 kg.
- Before you use the chair make sure that the chair operates smoothly.
- If you detect a problem, make sure to repair or adjust the chair. Your supplier can help you to find and correct the problem.
- Make sure batteries are charged.
- Be aware that there is a point during every transfer when the wheelchair seat is not below you.
- Use extra care when you drive your chair in reverse. You may lose control or fall if one of the rear wheels hits an object.
- Never use your chair on a slope unless you are sure you can do so without losing traction.
- Lift the chair only by non-detachable parts of the main frame.
- Never make direct contact both terminals, as an explosion may occur.
- Do not use this chair if any of the tires are under- or over-inflated.
- When using cell phones you should switch off the chair because of causing electromagnetic radiations.

B. CHARACTERISTICS (Fig. 1)

Back upholstery	1	Castors	9
Armrest	2	Footrests	10
Control handle	3	Cushion	11
Height and width adjustable armrest	4	Control	12
Stabilising bar	5	Joystick	13
Battery boxes	6		
Drive wheel	7		
Motors	8		

C. ASSEMBLY

1. Footrest (Fig. 2): Press the release button in order to be able to swivel a footrest right or left. To remove it from the wheelchair, press the button, rotate the footrest approximately 90°, and pull upward.

2. Armrest (Fig. 3): Insert the tubes of the armrest in the receptor tubes and turn the handle right to fix the position.

3. Anti-tip wheels: These are mounted when the wheelchair leaves the factory. The anti-tip wheels tube is fitted to the frame tube by a mounting pin. They will be removed by pressing this pin inwards. Reverse the procedure to assemble. If anti-tips wheels are fitted, the wheelchair may interfere with kerbs when mounting or dismounting.

4. Kerb climber (Fig. 4)

- Put the connecting brackets in each lower frame tube, and hold them slightly with the screw and the nut, using a 5 mm. Allen key. (1)
- Insert the supports in each bracket. (2)
- Put the kerb climber in the supports.
- The fastening lops are inserted through the holes of the kerb climber tube and the holes of the supports. (3)

5. Batteries:

- ❑ Lean the side supports of the battery housings on the rails.
- ❑ Slide the housings forward on the rail until they are hooked at the anchoring points. Place at the outermost position the battery housing that has access to the connector, and point it toward the side of the wheelchair on which the control is situated, so that access is facilitated for connecting the battery to the control.
- ❑ Interconnect the two batteries.
- ❑ Once you have made sure that the connections are secure, connect the outermost battery to the control. To remove the battery housings, disconnect the battery from the control, disconnect the batteries from each other. To remove the battery housings, push them upward until they are free of the anchoring points. Slide them outward on the rail.

6. Wheels: If it is necessary to remove one of the wheels (e.g. in order to repair them or in case of puncture), proceed as follows:

Front wheel

1. Use a 7 mm. Allen wrench to remove the fork screw.
2. Remove the damaged wheel.
3. When the wheel has been repaired, follow the reverse procedure.

Back wheel:

1. Using a screwdriver, remove the cover of the hub.
2. Loosen this screw with a 19 mm. hexagonal socket wrench.
3. Remove the damaged wheel. When it has been repaired, follow the reverse procedure.

7. Seat belt: In case the user needs a seat belt, this can be fastened to the frame as shown in the picture 5

Note: This wheelchair is designed to be repaired and assembled by the dealer and not the end user. The end user has to disassemble and assemble the chair only for transportation (D. Use: folding for transportation).

D. USE

1. Handling the wheelchair

Folding for transportation: First remove the batteries as described in Chapter C. 5. and make sure that the footrest platforms are in vertical position. Take hold of the seat upholstery by the middle and pull it upward until the unit is fully folded. To lift the batteries please use the handles on the lid of the battery boxes. To lift the chair when folded please use the upper tubes of the side frames and the push handles.

Opening: Pull downward on the upper tubes of the cross until the wheelchair is completely open. Put the batteries in place.

Note: Never lift the wheelchair by the armrests, since they are detachable and harm could be done to the user or to the wheelchair.

Reclining back (4° - 55°) (Fig. 6): By pulling the 2 levers at the same time the backrest is released and can be set to the required position. Releasing the 2 levers will lock the backrest automatically in the new position. To bring the backrest back to the upright position, just move it (no release necessary) to the required position.

Caution: The reclining back is only to be used in combination with the active anti-tips setting

Caution: The reclining back is only to be used in combination with a stabilizer bar. Please always make sure that the stabilizer bar is locked in place on both sides.

Caution: Before folding the wheelchair, make sure that the stabilizer bar is removed

Caution: In order to avoid any reduction in the cable function, you must make sure that the cables are not completely taut.

Caution: When adjusting the back angle, be careful not to get your fingers caught.

Caution: Before adjusting the back angle, put the anti tip tubes in to the active position

Elevating legrest: **To elevate:** Lift your lower leg and press the release button. Pull the legrest upwards and adjust it to the desired height. As soon as you release the button, the legrest will click back into place.

To lower: Lift your lower leg and press the release button to unlock the legrest to lower it. As soon as you release the button, the legrest will click back into place.

CAUTION: The elevating legrest is not allowed to be adjusted into the lowest position to avoid any interference with the casters. CAUTION: Keep hands clear of the adjustment mechanism between the frame and the movable

parts of the legrest while elevating or lowering the legrest. CAUTION: Legrests are not to be used for lifting or carrying the wheelchair with an occupant.

To steer the chair simply move the joystick or knob in the desired direction. The further forward you push it, the faster the wheelchair will travel. The knob regulates the power and speed. When it is released, the brakes go on automatically and the wheelchair is stationary. It is important that you stop the wheelchair before you change direction from forward to reverse. Always turn off the wheelchair before stepping out of it. Look ahead while you drive, not at the controls. Remember that only with practice will you become a competent driver.

Joystick: Your F35 is controlled by a joystick, which determines speed and direction. The more forward the joystick is pushed, the faster the wheelchair travels. When the joystick returns to its central position, the brakes enter into operation automatically. On flat ground, braking distance is equal to the length of the wheelchair. On a hill it may be a little more, and you will note how the brakes go on automatically when the wheelchair is almost stationary.

Other controls (Fig. 7)

- On/off button: This connects or disconnects the entire electronic system, which supplies energy to the motors. Do not use this button to stop the chair except in case of emergency, in doing so may damage the wheelchair.

- Battery Gauge: This indicates that the wheelchair is under power. It also indicates any fault there may be in functioning. The number of lights flashing indicates the fault type. (See "Battery Gauge" in this manual.)

- Speed indicator: This indicates the maximum speed set for the wheelchair. There are five speeds predefined. The first is the lowest, the last is the highest.

- Speed decrease button: Decrease the maximum speed setting

- Speed increase button: Increase the maximum speed setting

- Horn switch

- Battery Gauge

- If it remains lit, this means that everything is OK. It flashes to indicate any fault that may have arisen. Slow and stable flashing means the batteries need charging.

- If it flashes once every 2.5 seconds, this means that the control system has deactivated particularly if the wheelchair has not been used for a certain period of time. To reactivate it, turn off the system and then turn it on again.

- If the fault indicator flashes rapidly, this indicates that there is a fault. Turn off the system. Check the state of the batteries and the connections, and turn on the system again. If the flashing continues, the number of bars flashing will indicate the location of the fault.

- If the number of bars on is 1: The batteries need charging, or are badly connected.

- If the number of bars on is 2: The left motor is improperly connected.

- If the number of bars on is 3: Electrical fault in the left motor.

- If the number of bars on is 4: The right motor is improperly connected.

- If the number of bars on is 5: Electrical fault in the right motor.

- If the number of bars on is 6: The battery charger is plugged in.

- If the number of bars on is 7: Fault in the control.

- If the number of bars on is 8: Fault in the control system. Make sure the control system connections are secure.

- If the number of bars on is 9: Fault in the electromagnetic brake.

- If the number of bars on is 10: Excess voltage.

Safety switch: In the unlikely event that the wheelchair is overloaded, there are safety devices built into the electronic system. The power of the motor will be automatically reduced. This device cuts in only in extreme circumstances, such as repeated climbing of steep hills in very high ambient temperatures. The button on the side of the battery box immediately trips, and the wheelchair stops. This safety switch may function again when the button is again pressed. Within a few seconds you will be able to resume normal speed.

Slopes and ramps: The following instructions explain how to manoeuvre correctly on steep ramps and slopes, especially in the course of descent. If you stop on an incline, start off slowly and, if necessary, lean forward to counteract the tendency of the front wheels to rise. On descents it is important not to allow the wheelchair to exceed normal speed. Indeed, the safest course is to go down steep hills slowly and to stop immediately if you have any doubts about the steering. If the chair gathers speed, move the joystick to its central position to slow down, or stop completely. Then start off again slowly and do not allow speed to increase.

Note: Make sure that the automatic brakes are engaged (freewheel device) when climbing or descending a slope.

Transfers: Always position the wheelchair close to the place where the transfer will be carried out. It is important that the controller is turned OFFs, so that the wheelchair will not move throughout the procedure. Remove the armrest. The wheelchair is highly stable owing to its weight. Do not stand up on the footplates at any time during the transfer.

Freewheel device (Fig. 8)

- Turn outwards the lever on the back of each motor so that each drive wheel is in freewheel position and will turn freely. If you wish to push the wheelchair with the motors off, you should follow this procedure with both wheels.
- When the levers are turned inwards, the drive wheel will automatically connect with the propulsion mechanism. Follow this procedure with both wheels.

2. Batteries

24V (2x12V) / 40 Amph. maintenance free

Connector: 3 pins “Cannon “ type

Dimensions: 197x165x170 mm.

Charger: 24VDC 10 Amp

Capacity at the discharge time: 34 Ah/5h.

Charging of batteries(Charger Specification): Only use chargers for cyclic batteries, Gel-Type with 24V/10A output nominal max.: To charge batteries, connect the charger to the 230V (+15-10%), 50Hz mains, and then to the control.

Operation: When the charger has been connected to the power supply the red light will go on. Connect the charger to the control and then the yellow light will illuminate. When the green light illuminates or flashes, this will indicate the charging is complete. If the charger is not connected to the control it does not provide charge current. It is possible that the charger’s metallic box may slightly raise its temperature due to use.

General care of batteries

- Avoid heavy discharges from the batteries. If you continue driving until the chair is virtually immobile, you will have reduced the useful life of the batteries.
- The batteries should never be left uncharged. A battery that remains uncharged for two weeks loses a certain capacity that cannot be recovered later.
- The batteries should always be charged completely.
- Your batteries should be charged only by means of the charger supplied with the wheelchair. The battery needs a “running-in» period over six/twelve charges; and will not attain to its maximum range until the end of this period. It is better to limit the use of the wheelchair and the length of your trips until the batteries have been totally “run in».
- It is advisable to charge batteries every day, after using the wheelchair.
- Range: Batteries give the chair a range of approximately 25 kilometres, depending on the weight of the user and the terrain.
- Do not leave the batteries exposed to environmental conditions such as rain, humidity, high temperatures or fire.
- Check the conditions of the batteries and their connections to prevent from risks e.g. leaks

Safety guidelines for charging

- The charger is designed for indoor use only. Do not use outdoors or expose to rain, snow, spray or moisture.
- Suitable temperature for charging the batteries: -10°C up to +40°C
- The use of an accessory that has not been sold by Sunrise Medical, or that does not have Sunrise Medicals approval, may result in the risk of fire, electrocution, or personal injury.
- Make sure the cable is laid out in such manner that it cannot be stepped on, that it cannot be tripped over, and that it is not exposed to damage or stress.

- ❑ Extension cords should not be used unless there is no alternative. The use of the wrong extension cord could cause fire or electrocution. If an extension cord must be used, make sure the pins of its plug are equal in number, size, and shape to those of the charger plug, that the cable is correctly connected.
- ❑ Do not use the charger if it has received a hard knock, been dropped, or if it has been otherwise damaged. Take it to a qualified technician.
- ❑ Do not disassemble the charger. For repair it should always be handed over to the manufacturer. The wrong installation could lead to fire or electrocution.
- ❑ Never place the charger directly on top of the battery being charged.
- ❑ Never leave the charger on a rug or soft surface. Always place it on a hard surface.
- ❑ Due to an electronic circuit of the charger, the batteries can be permanently connected without any harm.

IMPORTANT

- ❑ Never smoke, and never allow the generation of sparks or flames, in the vicinity of battery or charger.
- ❑ Take special care to ensure that metal tools cannot fall on the battery, since a spark or short circuit could result in the battery or other electrical component that would cause an explosion. Before working on the battery, remove all personal effects that are metal and loose fittings from your own.
- ❑ When buying spare batteries, only use types which are approved by Sunrise Medical. If in doubt, consult your Sunrise Medical service agent.

3. Tyre pressure: It is important to check inflation pressure regularly, along with the state of wear of the tyres. The maximum pressure recommended for the front wheels is 36 psi (2.5 bars), while for the rear wheels it is 40 psi (2.7 bars). Caution: never inflate the tyres with a service station air pump. It is recommended that you use a manual pump or a pressure regulator (manometer).

4. Kerb climber

Climbing kerbs

1. Go forward to the kerb at a low and uniform speed.
2. When the Kerb Climber meets the kerb, the wheelchair should be moving slowly. Low kerbs can be climbed from a stationary position.
3. Allow the motors sufficient power to raise the front of the chair so that it will be supported on the kerb. Then release a little more power and increase speed so that the driving wheels climb the kerb smoothly and without interruption. As far as possible, hold the joystick in forward-drive position.

Descending from kerbs

1. Approach slowly and carefully in reverse until the rear wheels are at the edge.
2. As slowly as possible, descend from the kerb in reverse and on the rear wheels. You will feel more secure if you lean forward, but do not worry if you are unable to: the wheelchair is extremely stable.
3. The front part of the chair will in turn descend naturally from the kerb as the chair continues away from it in reverse. It may help, in a difficult situation, to lean back while the front ascends, and lean forward while the rear wheels rise. However, too much movement in the seat at the wrong moment may make the situation worse, for which reason it is better not to change position more than necessary.

E. ADJUSTMENTS

1. Adjustment of the footrest (Fig. 9) To adjust the height of a footrest, remove the securing nut from the screw in the tube. Remove the screw, set the footrest at the height desired, and replace the screw in the hole corresponding to that height. Secure the screw with the nut. For these adjustments it is recommended you use a 10 mm. spanner. Make sure the screw and nut are tightened securely.

2. Adjustment of the armrest

Height adjustment: (Fig. 10): Turn the handle to the left to free the tube of the armrest. Raise as you wish, and then turn the handle to the right to fix the position.

Width adjustment: (Fig. 11) Loosen the two hexagonal screws in the tube under the armrest with a 10 mm. hexagonal wrench. Move the armrest in or out until you have the position you want. Tighten the screws securely.

3. Control Adjustment (Fig. 12): You can slide the control along the rail and set it as you wish. To adjust the control arm rotate the handle, located under the rail to the left. Turn to the right to fix the position.

4. Programming the control. When the chair leaves the factory, the parameters of the controller are set to default.
Note: SUNRISE MEDICAL does not accept responsibility for damages which result from unexpected stopping of the wheelchair or inappropriate programming or unauthorised use of the wheelchair.

Note: Programming the controller of the wheelchair is only allowed through authorised Sunrise Medical agents. Wrong controller settings could cause driving outside the safe limits and could cause damage and injury.

F. CONTROLS AND MAINTENANCE

	Daily	Weekly	Monthly	Quarterly	Annually
Charge battery	X				
Check battery level indicator	X				
Ensure all removable parts are securely fastened	X				
Check all the electrical connections		X			
Wipe vehicle with a damp cloth		X			
Check tyre pressures		X			
Check tyres for wear			X		
Clean and protect the battery terminals with petroleum jelly				X	
Check condition of upholstery				X	
Complete inspection and service by an authorised supplier					X

G. SPECIFICATIONS

	Min.	Max.		Min.	Max.
Effective seat width	430 mm.		Seat plane angle	5°	
Effective seat depth	420 mm.		Backrest angle	8°	
Backrest height	470 mm.	495 mm.	Leg to seat surface angle	70°	
Overall length (w/legrest)	1060 mm.		Max. Speed forward		6 km/h.
Overall width	600 mm.	635 mm.	Obstacle climbing (w/kerb climb.)	50 mm.	100 mm.
Climbing ability upwards		10°	Max. safe slope		10°
Seat surface height at front edge	510 mm.		Min. Turning radius	900 mm.	
Footrest to seat distance	420 mm.	490 mm.	Max. User weight		120 kg
Armrest to seat distance	180 mm	300 mm	Mass of the heaviest part (frame without detachable parts and batteries)		29 kg.
Folded length	770 mm				
Folded width	350 mm.				
Folded height	940 mm.				
Total mass (w/batteries)	63 kg.				

H. GUARANTEE

SUNRISE MEDICAL guarantees this product for two years as from the date indicated on the invoice, receipt or delivery note. For products containing batteries, the battery warranty is 6 months. This warranty covers parts in need of replacement, labour and transport.

In warranty claims the user must submit the product and the purchase invoice as proof of delivery date.

This warranty does not cover:

- NORMAL WEAR AND TEAR, e.g. on upholstery, tyres, inner tubes, rims, wheels, fabric, etc.
- ACCIDENTS, e.g. bent structures, scratches, flat tyres, etc.
- IMPROPER USE and damage caused by users who weigh more than the weight limit indicated in the user's manual.
- LACK OF MAINTENANCE and failure to follow the conservation instructions in the user's manual.
- MODIFICATIONS or REPAIRS carried out by persons not authorised by Sunrise Medical.
- PRODUCTS WITH NO INVOICE OR SERIAL NUMBER (when appropriate).

TAKE CARE OF THE ENVIROMENT; FOLLOW YOUR TOWN'S COUNCIL LAWS TO GET RID OF YOUR USED BATTERIES AND WHEELCHAIR.

I. SERVICE HISTORY

This section is designed to assist you in keeping a record of any service and repairs to your wheelchair. The service Agent will complete this section and return the manual to you.

Model					Serial No.				
Year	1	2	3	4	Year	1	2	3	4
Service dates					Service dates				
Controller					Chassis				
On/Off switch					Condition				
Output plug					Steering				
Operation					Upholstery				
Dynamic braking					Seat				
Programmable configuration					Back				
Batteries					Armrests				
Levels					Electric's				
Connections					Condition of loom				
Discharge tests					Connections				
Wheels/Tyres					Test run				
Wear					Forward				
Pressure					Reverse				
Bearings					Emergency stop				
Wheel nuts					Left run				
Motors					Right run				
Wiring					Up/down slope				
Noises					Over obstacle				
Connections					Parking brake				
Brakes									
Brushes									

NOTE: Only use Sunrise Medical parts for service and repairs

Apart from the Service History I. and part C. Assembly in this handbook, there is no service manual available. Replacement units are available on request by the dealer for repairing or changing. Please refer to the F35 parts manual available.

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