## RECICLAJE

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P

## INTRODUCTION

Congratulations for the purchase of SIGEL Liquid Flotation cushion designed and made so that you enjoy ideal performances, comfort and personal satisfaction.

The SIGEL cushions for pressure ulcers provide a larger contact area, which reduces the pressure per unit of area, one of the factors that influence the appearance of pressure ulcers. They also provide refreshment thanks to the special liquid inside.

Every cushion SIGEL is made of an internal part and a fireproof cloth cover which superior part complies with the norms JOINS IN 1021-1y-2 and it is classified as "M1-No Fireproof" according to the norm JOINS 23721-27, and the lower part is made of an anti-slip cloth that prevents the cushion from moving in the seat when being used.

The cushions SIGEL comply with the norm prEN 12182: "Technical helps for persons with disability. General requirements and test methods."

Product in accordance with the European Board of Sanitary Products 93/42/CEE.

## RECOMMENDATIONS

- Ensure that the patient' weight is lower than the maximum weight that can support the cushion.
- Check that the dimensions of the cushion are adapted to your wheelchair.
- Place the cushion on the wheelchair with the anti-slip cloth at the bottom.
- Eliminate from the superior part of the cushion the possible creases that could appear.
- Check regularly the state of the internal part and of the cover after several washings.
- Avoid the accumulation of humidity in the case.
- Avoid putting the internal part close to any fire sources.

UK

## GUARANTEE

Your SIGEL Liquid Flotation pressure ulcer cushion has been rigorously inspected; on the label of the internal part you will find a serial number which relates the above mentioned cushion to the materials used and the manufacturing processes. This cushion has guarantee of 1 year from the date of purchase in normal conditions of use and respecting the recommendations of use.

In the event of a defect of the cushion, please contact the distributor where it was purchased, with the written warranty and the proof of purchase, so that the cushion may be inspected and if it is necessary, in accordance with the manufacturer, be replaced or repaired. The distributor and VIRMEDIC S.L. shall be excluded from any responsibility in the event of imperfections as a result of misuse or failure to adhere to the recommendations given in these instructions. (Puncture, rip, etc.)

## MAINTENANCE

For hygienic reasons it is recommended that the cushion is always used by the same person. The internal part can be washed by introducing it in a container filled with water and neutral soap, proceeding to its brushing and later rinse with water. The drying will be done with a cloth or towel or letting it dry to the air. It will be done in horizontal position in a fresh and dry place, avoiding the exhibition to the solar beams. Do not store on sharp objects or sharp edges that could damage the cover or internal part. In the event of making a puncture in the internal part, dry and clean with alcohol the area around the puncture and put a few stickers or cello tape for soft plastic, making it thicker in the centre (we recommend to use Vinilceys® and then wait 24 hours before using it again)

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