Invacare® Sonata

en	Bed	User Manual.	3
es	Cama	Manual del usuario	17
fr	Lit	Manuel d'utilisation.	33
it	Letto	Manuale d'uso	47
pt	Cama	Manual de utilização	63

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future
reference.

Yes, you can.®

© 2015 Invacare® Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

This bed has been tested and approved according to IEC 60601-2-52.

This bed has undergone a risk analysis according to EN ISO 14971.

Invacare is certified according to ISO 9001 and ISO 13485

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Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

I General	4
1.1 About this manual	4
1.2 Intended use	4
1.3 Service life	4
1.4 Warranty	4
2 Safety	5
2.1 General warnings	5
2.2 Labels and symbols on the product	5
3 Setup	7
3.1 Receiving the bed	7
3.2 Main parts of bed	7
3.3 Assembly of mattress support	7
3.4 Electrically operated mattress support	7
3.5 Mounting of bed ends and legs	8
3.6 Wiring	9
4 Operating the bed	10
4.1 Operating the castor brakes	10
4.2 Side rails	10
4.3 Operating the hand control unit	10
4.4 Operating the bed functions manually	10
4.5 Emergency release of the backrest or thigh/leg section	11
4.6 Mattress retainers	11
4.7 Lifting pole	11
5 Accessories	12
5.1 List of available accessories	12
5.2 Mattress	12
6 Maintenance	13
6.1 Maintenance schedule	13
6.2 Cleaning	13
7 After Use	14
7.1 Waste disposal	14
8 Troubleshooting	15
8.1 Troubleshooting electrical system	15
9 Technical data	16
9.1 Dimensions	16
9.2 Weights	16
9.3 Environmental conditions	16
9.4 Electrical system	16

I General

I.1 About this manual

This user manual contains important information about the handling of the product. In order to ensure safety when using the product, read the user manual carefully and follow the safety instructions.

All indications of right and left are based on a user lying on his back in the bed.

Please note that there may be sections in this user manual, which are not relevant to your product, since this manual applies to all existing modules (on the date of printing).

If a problem should arise in connection with the delivered product, please contact your *Invacare*® supplier. An address list is shown on the back side at this manual.

Symbols in this manual

In this user manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

I.2 Intended use

The bed has been developed for domestic care and care in long term care facilities. The bed features a comfortable sitting and lying position for the patient. Furthermore, ergonomic operation for the carer is ensured.

The bed is defined to be used according to:

- *Application environment 3*; Long-term care in a medical area where medical supervision is required and monitoring is provided if necessary and medical electrical equipment used in medical procedures may be provided to help maintain or improve the condition of the patient.
- *Application environment 4*; Care provided in a domestic area where medical electrical equipment is used to alleviate or compensate for an injury, disability or disease.

The bed is not intended for children under 12 years and for psychiatric patients.

The bed is not intended for transporting the patient.

WARNING!

Any other or incorrect use could lead to hazardous situations.

Invacare accepts no liability for any use, change or assembly of the product, other than stated in this user manual.

I.3 Service life

The expected service life of this product is seven years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

I.4 Warranty

The warranty covers all material and production defects for two years from the date of delivery, provided it can be demonstrated that such defects were present before delivery. All manufacturing faults or defects must be promptly reported.

Invacare® may repair the fault or replace the component. The warranty provided by *Invacare*® does not cover additional costs (transport, packaging, labour, sundry expenses, etc. are for the customer's account).

The warranty does not cover:

- Damage caused during transport that is not directly reported to the forwarder at the moment of delivery.
- Repairs performed by unauthorized centers and personnel.
- Parts subject to normal wear.
- Malicious damages or damaged caused by improper use of the bed.

2 Safety

2.1 General warnings

WARNING!

Risk of entrapment / suffocation

There's a risk of entrapment / suffocation between mattress support, side rail and bed end

- The bed must not be used by patients under 12 years of age, or by patients with body size equivalent to an average 12 years old or smaller.
- The bed, in combination with side rails must not be used for persons under 45 kg or under 150 cm height. Either for persons that are restless (spasms) or confused, unless a professional risk assessment has taken place and been accepted.

WARNING!

Risk due to electromagnetic interference

Electromagnetic interference between the bed and other electrical products can occur.

- To reduce or eliminate such electromagnetic interference, increase the distance between the bed and the products or switch them off.

This medical bed can be used together with medical electrical equipment connected to the heart (intracardially) or blood vessels (intravascularly) provided that following points are respected:

- Medical electrical equipment should not be fixed on the bed's metallic accessories such as side rails, lifting pole, drip rod, bed ends, etc.
- The medical electrical equipment power supply cord should be kept clear of the accessories or any other moving parts of the bed.

WARNING!

Risk of patients slipping through the openings

The bed fulfils all requirements regarding maximum distances. However, it is possible that patients with small body dimensions slip through the openings between the side rails or through the opening between the side rail and the mattress support.

- Pay special attention, if the bed is used for the care of patients with small body dimensions.

CAUTION!

- There is a risk of entrapment of fingers in the bed moving parts. Pay attention to your fingers.
- Always lower the bed to the lowest position before leaving the patient in the bed unattended.
- When entering or exiting the bed, use the height adjustment. Use the backrest as support. The thigh and leg section must be horizontal, otherwise there is a risk of overloading the mattress support.


IMPORTANT!

- Make sure that there is nothing under, over, or near the bed that can limit the movement of the bed or the mattress support, such as furniture, window frames and storage boxes.
- Do not roll over the mains cable. Do not bring mains cable into moving parts.
- Disconnect the plug from the mains before moving the bed.
- Make sure that no cables (mains or from other equipment) are jammed or damaged, when the bed is used.
- Do not place the bed close to a direct heat source (fire place, radiator etc.) and not in direct sunlight.

2.2 Labels and symbols on the product

The product label is placed on the frame of the bed.

Symbols on the label

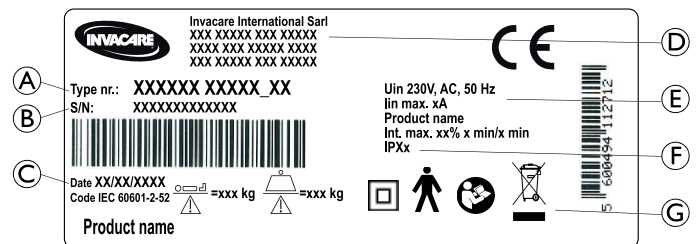A	Type number of the bed
B	Serial number
C	Date of production
D	Manufacturer; Name and Address
E	Electrical information
F	IPX Degree
G	Symbols

Explanations to abbreviations on label			
lin	Current in	Max	Maximum
Vin	Voltage in	min	minutes
Int.	Intermitens	kg	kilogram
AC	Alternating current	A	Ampere

Symbols on the product

All symbols might not be present on all products.

	Max. patient weight
	Max. safe working load
	Insulation class II
	Type B Applied Part.
	Potential equalization

	Refer to the user manual.
--	---------------------------

	Waste disposal and recycling information.
	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

3 Setup

3.1 Receiving the bed

When you receive the bed, check the packaging. If the bed shows any signs of damage upon delivery, please read Section Warranty.

WARNING!
Risk of personal damage

- There is a risk of entrapment or squeezing, while assembling or disassembling of the bed.
- The assembly of the bed and mounting of accessories must be done by authorized or trained personnel.
 - Follow instructions carefully.

IMPORTANT!

- To avoid condensation the bed should not be used until it has reached a temperature of 10-40° C.

IMPORTANT!

- After each assembly, check that all fittings are properly tightened and that all parts have the correct function.


WARNING!

- The standard electrical equipment must not be combined with electrical equipment that is not designed for this bed.

3.2 Main parts of bed

The bed can be configured either for manual or electrical operation.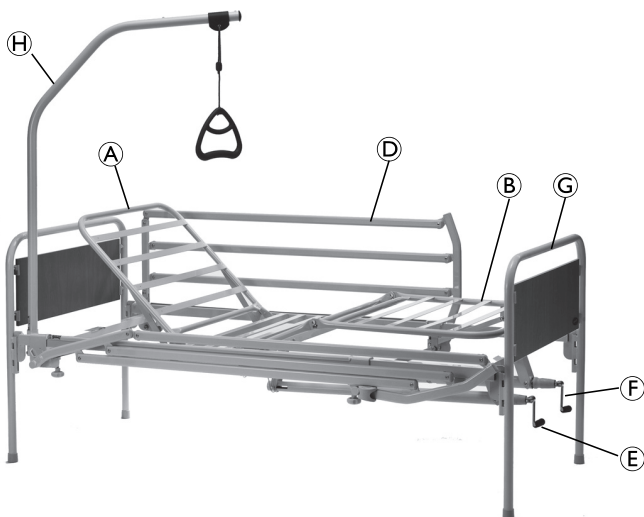

Scope of delivery:

- Ⓐ Mattress support, head section
- Ⓑ Mattress support, foot section
- Ⓓ Pair of side rails
- Ⓔ Motor / control box or crank for head section
- Ⓕ Motor or crank for thigh section
- Ⓖ Pair of bed ends
- Ⓗ Lifting pole (optional)
Hand control

2 pcs. of mattress support retainers (2-section beds)

4 pcs. of mattress support retainers (4-section beds)

3.3 Assembly of mattress support

CAUTION!

Risk of injuries or finger pinch

- Keep your fingers away from movable parts during assembling, disassembling or adjusting.

IMPORTANT!

- Before any use and before operating the bed:
- Check that the bed is correctly mounted according to instructions.
 - Check that all screws are properly tightened.
 - Ensure that the pipe pins are fully inserted through both holes of the fixation and that the spring clip is completely locked.

After assembly of the mattress support; check:


- that all plugs of the motors and hand control are correctly connected to the control unit according to the printed symbols.
- that top frame, head- and leg end are completely pushed together.
- that the lock pin is engaged.
- that bed ends are correctly attached.

Sonata can have either a complete or a separated mattress support.

If your bed is equipped with a complete mattress support, proceed to 3.5 Mounting of bed ends and legs, page 8 .

Assemble the separated mattress support

1.

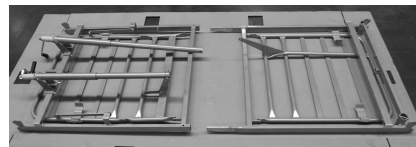
Turn the mattress support up-side-down on a flat, clean surface.

2.

Place the two inserts in the head end of the mattress support and push the ends together.

3.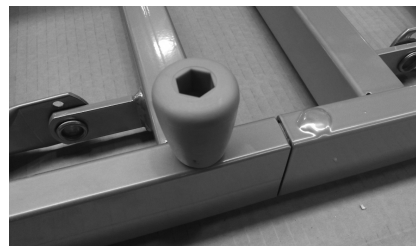


Tighten with the two thumb screws.

3.4 Electrically operated mattress support

The longer motor (assembled on the control box) operates the backrest.

The shorter motor operates the leg section.

1.

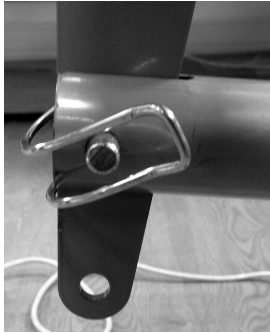Join the leg section tube with the leg section bracket, using a pipe pin in the upper hole of the bracket.

2.

Join the backrest tube with the backrest bracket, using a pipe pin in the hole of the bracket.


3.

Connect the section motor wires to the control box.

- Backrest motor to socket 1.
- Leg section motor to socket 2.

4.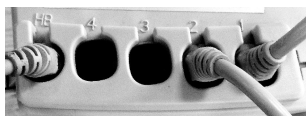

Attach the locking cam over all motor plugs.

3.5 Mounting of bed ends and legs

The bed ends can be equipped with different types of legs:

- Bed ends with integrated legs
- Height adjustable or fixed height legs

CAUTION!
Different height fixation of legs can cause instability in the bed

On the bayonet system, the brackets of the bed ends have two different fixations, for different heights. Legs with adjustable height can be fixed in three different heights.

- Before using the bed, make sure that all brackets/fixations are set to the same height on all four legs.

Bed end with integrated legs

Latch the mattress support to the bed ends and press firmly into position. Do this one end at a time.


(1) High position

(2) Low position

Make sure that the bed ends are on the same level. It is possible to level the bed end in the high position (1) or the low position (2).

Bed end with adjustable/fixed legs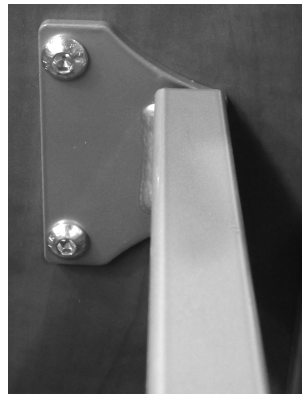

For Sabrina, Camile and Nadine bed ends; mount the brackets for the bed ends on the side tube of the mattress platform. Tighten the screws firmly with a 5 mm Allen Key.


Lower the Vibeke bed end in the U-bracket fitting. Press down firmly and ensure that bed end is fully seated at bottom of U-bracket fitting.

Mattress support with mountable legs

On the mattress support with the mountable legs, the legs are mounted with screws to the mattress support.

1.

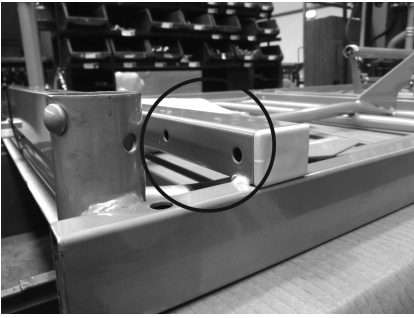Place the bracket on the cross tube.

2.

Insert the screws and tighten the bolts.

Height adjustable legs

The legs can be adjusted in 3 different heights with a pipe pin. Ensure the pipe pin is fully inserted through both holes and that the pipe pin is completely locked. Make sure that all four legs are adjusted to the same height.

WARNING!

- The Diana side rail can not be used on the bed when the bed is configured with adjustable legs without castors and when the legs are mounted in the lowest height.

3.6 Wiring

IMPORTANT!

- Not correctly drawn wires can be damaged when operating the sections with the motors.
- All wires must be kept clear of the floor and must not block the castors.
 - All wires (motors, hand control and mains) must be connected to the control box
 - The locking cam must be attached over the plugs in the control box.

Connect the mains wire plug to the 230 V socket.

4 Operating the bed

4.1 Operating the castor brakes

Each of the castors is provided with brakes for both lengthwise and crosswise locking. The brakes are foot-operated.

When the bed is positioned correctly, at least one castor at the head end and one castor at the foot end must be braked.

WARNING!

Risk of injury


- The bed's castors must be locked whilst nursing a patient in bed or when using any positioning function.
- The bed's castors must be locked when the patient moving in or out of the bed.

CAUTION!

Risk of trapping

- Do not release the brake with the fingers.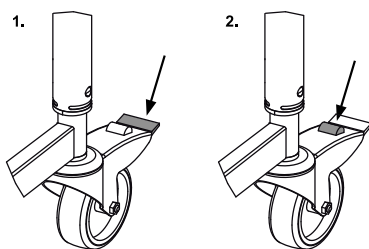

1. For braking, step on the pedal
2. For releasing the brake, step on the release pedal

Set-off from the castors might under special conditions appear at different types of absorbing floor covering including untreated or badly treated floors. In matters of doubt, it's recommended to place a suitable kind of protection between the castors and the floor.

4.2 Side rails

WARNING!

Risk of entrapment or suffocation

There's a risk of entrapment or suffocation between mattress support, side rail and bed end.
- Always ensure correct fitting of the side rails.


WARNING!

Risk of entrapment

There's a risk of entrapment or suffocation between mattress support, side rail and bed end.
- When using side rails, always make sure that the distance between the bed end and the side rail's handle upper edge is less than 6 cm in the head end and more than 32 cm in the foot end of the bed.

A sticker is placed on the side rail to indicate the minimum space between side rail and the head end of the bed: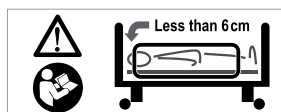

CAUTION!

There's a risk of entrapment of fingers on assembly and operation of the side rail.

Mounting, dismounting and operating


The side rail must be mounted according to the separate mounting instruction for the chosen side rail.

CAUTION!

If the side rail is not locked properly it can fall down.
- Pull/push the top bar of the side rail to ensure that the locking mechanism is properly engaged.

4.3 Operating the hand control unit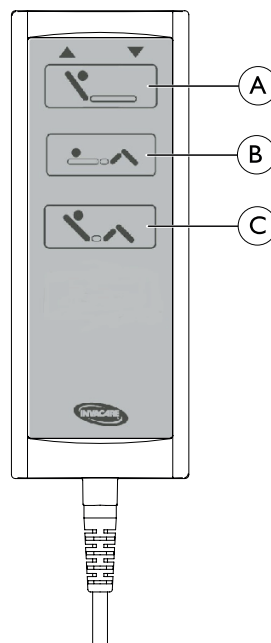

- ▲ Move up (press left side of corresponding button)
- ▼ Move down (press right side of corresponding button)
- Ⓐ Backrest adjustment
- Ⓑ Thigh section adjustment
- Ⓒ Sitting position adjustment

1. Use the buttons on the hand control to operate the electrical functions of the bed.

4.4 Operating the bed functions manually

If the bed is configured for manual operation, it's not equipped with a hand control unit, but with a crank, positioned at the foot end of the bed.

The two-section bed has one crank to operate the head section. The four-section bed has one crank to operate the head section and one crank to operate the thigh section.

1. UP: Turn the crank handle clockwise
2. DOWN: Turn the crank handle counterclockwise

WARNING!

Do not use the crank if an extra weight, apart from the user, is placed on the bed.

1.

When the back-/footrest has been height adjusted the crank needs to be locked.

2.

Lock/secure the crank by pushing the white sleeve inwards and bending the crank by the knuckle.

IMPORTANT!

Twisting or rotating the white sleeve will damage its internal components and the locking function will seize to work.

- Only push the white sleeve inwards without twisting.

3.

The handle always has to be in the folded position, when the crank is not used.

4.5 Emergency release of the backrest or thigh/leg section

An emergency release of the backrest or thigh/leg section could be necessary in the case of power- or motor failure.


CAUTION!
Risk of injury

- A minimum of two persons are required to release a mattress section.
- An emergency release of the height adjustment is NOT possible.
- Remove the plug from the mains socket before emergency release of the mattress support.

1. Both persons hold the mattress section.

2.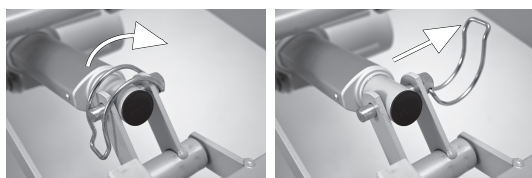

One of them pulls out the cotter pin (from the motor in question).

3. Both persons slowly lower the mattress section until it is completely down.

4.6 Mattress retainers

WARNING!
Risk of injury

If the mattress retainers are removed the mattress will not follow the movement, when adjusting the head-/legsections. The mattress can slip sideways and cause the user to fall out of the bed or get trapped in the bed.

- Always use mattress retainers and be careful to put them back after any adjustments.

4.7 Lifting pole

WARNING!
Risk of injury

If the lifting pole is used while the handle has been turned away from the bed, the bed can tip when the handle is used.

- Position the lifting pole in such a way that the handle extends inwards across the bed.

Mounting

1. Remove the plug from the lifting pole tube at the head end of the bed where the lifting pole is to be placed.
2. Insert the lifting pole into the lifting pole tube and fix with the thumb screw.

Adjusting the height of the lifting pole handle

1.

3.

1. Loosen the cord.
2. The lifting handle can now be adjusted to the desired height.
3. Press the cord together and check that the cord is locked in the cord lock by pulling the handle.

The distance between the lifting pole and the mattress

Minimum (at 16 cm mattress)	36-59 cm
Maximum (at 10 cm mattress)	42-65 cm

5 Accessories

5.1 List of available accessories

- Side rails
 - Side rail Verso II
3/4 length collapsible steel side rail (156 cm x 40 cm), mounted on the side tube. Release button in the foot section
 - Side rail Diana
3/4 length lowering side rail (153cm x 37 cm). Mounted on the side tube.
 - Side rail Flora
Full length wooden side rail, in cherry.
- Side rail cover for Verso II
- Lifting pole
- Holder for hand control
- Drip rod bar
- Electrical up-grade kit for the mattress support

! Important

- Use only original accessories and spare parts. Spare parts lists and extra user manuals can be ordered from *Invacare* or via homepage.

5.2 Mattress

WARNING!

Safety aspects regarding combination of side rails and mattresses:

- In order to obtain the highest possible safety level, when using the bed in combination with side rails, it is important to respect the minimum and maximum measures for mattresses stated in the table.
- To mitigate the possibility of the user crawling over the top of the side rail, with the risk of severe injury due to e.g. falling, a minimum vertical distance of 22 cm between the top of the mattress and the top of the side rail are to be respected. The maximum mattress height in combination with a given side rail is stated in the table.
- To prevent head entrapment between the inside surface of the side rail and the mattress, with the risk of e.g. suffocation, a safety maximum space has to be respected. When especially very thick mattresses or soft mattresses (low density) or a combination of these it is important to focus on this risk. The minimum width (and length) of mattresses in combination with a given side rail is stated in the table.

Two-section bed

Side rail:	Mattress measures (in cm):			
	Min. height	Max. height	Min. width	Min. length
Diana	13	15	85	190
Flora	12	17	85	190
Verso II	8	18	85	190

Four-section bed

Side rail:	Mattress measures (in cm):			
	Min. height	Max. height	Min. width	Min. length
Diana	13	15	85	190

Flora	12	17	85	190
Verso II	8	18	85	190

Min. density: 38kg/m³

6 Maintenance

6.1 Maintenance schedule

- !** **Important**
– Service and maintenance of the bed must only be performed by personnel who have received the necessary instruction or training.

With normal daily use, service must be carried out according to the service schedule after two years of use and thereafter every second year.

When relocating the bed, service must be carried out according to the service schedule.

Before Use	Ensure that all manual and electrical parts are functioning correctly and are in a secure state.
After three month	Ensure that all manual and electrical parts are functioning, and tighten bolts, screws, nuts, etc.
Every two years	Service should be performed according to the maintenance chart. See below.

Bed / Id. no.:	Date / Initials				
Visual inspection of all parts of the bed (plastic deformation and/or wear and tear of welded joints)					
Control all centres of rotation (motors + mattress support parts)					
Check that all motors are running without failures (with regular speed and at low noise)					
Visual inspection of all bed ends for damages.					
Check that mains cable and plug are intact. Check the running of the cable.					
Check all remaining cables for damages. Also check the running of the cables.					
Check of siderails' fixing and locking/movement.					
Check the wheels (security, braking and free rolling)					

Lubrication plan

We recommend lubricating the bed according to the following instructions:

Lubricate with medically clean oil, e.g. KEN-WO 50:

- Points of rotation in mattress support and base frame - lubricate with oil.
- All of the motors' tension rod - lubricate with oil.
- All motor bearings - lubricate with oil.

 The wooden side rails gliding system must not be lubricated with oil - otherwise the wooden bars will move sluggishly.

6.2 Cleaning

- !** **Important**
– The bed does not tolerate cleaning in automatic washing plants or the use of water-jet based cleaning equipment.
– Make sure that the power plug is removed from the main socket before cleaning.
– Make sure that all plugs on the bed are in place, during washing.

- !** **Important details for cleaning IPX6 hand controls**
– Water may be pressurised, but never use high-pressure cleaning or steam cleaning directly on the electrical parts.

- The bed should be washed down using a wet sponge, cloth or brush.
- Use ordinary household cleaning agents. Never use acids, alkalines or solvents such as acetone or cellulose thinner.
- The backrest and the leg/thigh sections can be lifted from the mattress support, which facilitates cleaning and access to the control unit.
- The hand control, motors and control unit may be washed with brush and water.
- Dry the bed after cleaning.

7 After Use

7.1 Waste disposal

 Waste disposal/recycle must comply with the laws and regulations for waste handling in each country.

Invacare® is continuously working towards ensuring that the company's impact on environment, locally and globally, is reduced to a minimum.

We comply with the current environment legislation (e.g. WEEE and RoHS directive).

We only use REACH compliant materials and components.

- All wooden parts must be dismantled and sent for incineration.
- All electric parts must be dismantled and be disposed of as electric components.
- Plastic parts must be sent for incineration or recycling.
- Steel parts and castors must be disposed of as waste metals.

IMPORTANT!

Accumulator back-up

- Old accumulators must be returned to *Invacare®* or recycled as car batteries

8 Troubleshooting

8.1 Troubleshooting electrical system


Symptom	Possible cause	Remedy
Mains indicator does not light up	Mains are not connected	Connect mains
	Fuse in the control unit is blown	* Replace the control unit
	Control unit is defective	* Replace the control unit
Mains indicator lights up, but the motor is not running. The relay in the control unit makes a clicking noise.	Motor plug is not fully inserted into the control unit.	Insert the motor plug properly into the control unit
	The motor is defective.	* Replace the motor
	Motor cable is damaged.	* Replace the cable
	Control unit is defective	* Replace the control unit
Mains indicator lights up, but the motor is not running. No relay sound is heard from the control unit.	Control unit is defective	* Replace the control unit
	Hand control is defective	* Replace the hand control
Control unit is in order except for one direction on one channel.	Control unit is defective	* Replace the control unit
	Hand control is defective	* Replace the hand control
Motor is running, but the piston rod does not move.	Motor is damaged	* Replace the motor
The motor cannot lift full load.		
Motor noise, but no movement of piston rod.		
Piston rod operates inwards and not outwards.		
<p>* Service and maintenance of the bed must only be performed by personnel who have received the necessary instruction or training.</p> <p>Risk of personal injury and damage to the product. – The bed must be unplugged from the main power source before opening or repairing electrical parts.</p>		
<p> WARNING! Risk of personal injury and damage to the product. – The bed must be unplugged from the main power source before opening or repairing electrical parts.</p>		

9 Technical data

9.1 Dimensions

All measurements are stated in mm. All angles are stated in degrees. All measurements and angles are stated without tolerances.

Invacare® reserves the right to change the stated measurements and angles.

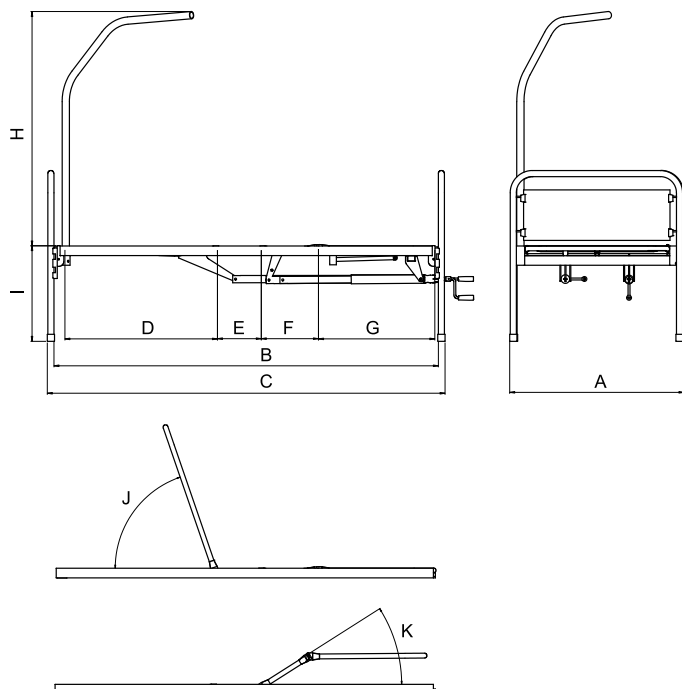A (outer)	890 – 1025 mm
B (inner)	2000 mm
C (outer)	2080 – 2130 mm
D	800 mm
E	230 mm
F	300 mm
G	610 mm
H	1230 mm
I	365 – 635 mm
J	0 – 72°
K	0 – 32°

9.2 Weights

	Max. patient weight (provided that the weight of the mattress and the accessories do not exceed 35 kg)	135 kg
	Max. safe working load (patient + accessories)	170 kg

Sonata standard bed, complete, excluding accessories	
Mattress support, head section four sections (2 pcs.)	15,5 kg

Mattress support, leg section four sections (2 pcs.)	14.0 kg
Mattress support, four sections (1 pc)	28,5 kg
Mattress support, four sections (1 pc)	22,5 kg
Side rail Verso II (1 pc.)	7.0 kg
Side rail Diana (1 pc.)	6.0 kg
Side rail Flora (1 pc.)	8,5 kg
Bed end Valeria (1 pc.)	4.9 kg
Bed end Camila (1 pc.)	11,0 kg
Bed end Sabrina (1 pc.)	6,8 kg
Bed end Nadine (1 pc.)	5,0 kg
Bed end Vibeke (1 pc.)	6,5 kg
Lifting pole	5,0 kg

9.3 Environmental conditions

	Storage and transportation	Operation
Temperature	-10°C to +50°C	+5°C to +40°C
Relative humidity	20% to 75%	
Atmospheric pressure	800 hPa to 1060 hPa	

Be aware that when a bed has been stored under low temperatures, it must be adjusted to operating conditions before use.

9.4 Electrical system

Voltage supply: $U_{in} = 220$ Volt, AC, 50 Hz (AC = Alternating current)
Maximum current input: $I_{in\ max.} = 1$ A
Intermittent (periodic motor operation): $I_{nt} = \max. 10\%$, 2 min / 18 min
Degree of protection: IPX6*
Insulation class: Class II
Type B Applied Part
Applied Part complying with the specified requirements for protection against electrical shock according to IEC60601-1. (An applied parts is a part of the medical equipment which is designed to come into physical contact with the patient or parts that are likely to be brought into contact with the patient.)
Sound level: 45 to 50 dB (A)

* IPX6 classification means that the electrical system is protected against water projected in powerful jets from any direction.

The bed is not provided with a mains switch, so the mains plug is the only separation from the mains.

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