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**ENGLISH****INSTRUCTIONS FOR USE AND PRESERVATION**

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

**SUITABLE FOR****ALL MODELS:**

- Prevention and auxiliary treatment of pressure sores.
- Help with postural changes.
- People who spend long periods of time seated.
- Postpartum pain relief and post-operative recovery in general.

**ONLY MODELS WITH HOLE (leave the injured area free of pressure):**

- Haemorrhoids.
- Fistulas and fissures in the anus.
- Postpartum fistulas and other painful processes.
- Post-operative recovery from diseases of the prostate or pelvic floor.
- Pain relief in the sacrococcygeal, ischiatic, gluteal, perineal (anal and/or urogenital) regions.

**NO MODEL RECOMMENDED FOR USE by people:**

- With no mobility.
- With excess weight or obesity.

**FITTING INSTRUCTIONS**

In order to achieve the highest degree of therapeutic effectiveness for different disorders and to prolong the service life of the product, it is essential to select the most suitable size for the patient or user (see the size and centimetre equivalence chart on the packaging).

**Follow the fitting instructions below:**

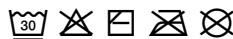
- 1- Can be used on a wheelchair or any type of seat (chair, armchair, etc.).
- 2- Place the cushion on the seat and attach to the rigid structure with the tapes.

**WASHING AND MAINTENANCE RECOMMENDATIONS**

Maintenance: regularly brush or shake the surface of the fibre to keep it in optimum condition and maximise the product's service life.

Washing: before washing, close any Velcro fasteners or zips.

Machine washable: machine wash with delicate programme and tepid water (30°C) with neutral detergent. Spin dry gently (400 rpm). For drying let it dry at room temperature. Do not use bleach. Do not hang up, iron or expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc. Do not dry clean.

When using or cleaning, do not use bleach, alcohol, ointments or liquid solvents. If the product is not well wrung out, detergent residue can remain and cause irritation to the skin and damage to the product.

Store in a dry place at room temperature.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

**COMPOSITION**

- > Fabrics:
  - Top fabric: 90% polyester, 10% acrylic with Sanitized treatment (contains biocidal substance: zinc pyrithione).
  - Bottom fabric: 65% polyester, 35% cotton.
- > Filling: 100% siliconised and conjugated hollow polyester fibre.
- > Tapes: 100% polyester.

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