# VERMEIREN

## Inovys II

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### **Instructions to the specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, November 2011

All rights reserved, including translation.

No part of this manual may be reproduced in any form whatsoever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2011

## Contents

<b>Preface</b> .....	<b>2</b>
<b>1 Productdescription</b> .....	<b>3</b>
1.1 Intended Use.....	3
1.2 Technical specifications .....	4
1.3 Drawing.....	5
1.4 Accessories.....	5
1.5 Explanation of symbols.....	5
1.6 Safety rules.....	6
<b>2 Use</b> .....	<b>7</b>
2.1 Carrying the wheelchair .....	7
2.2 Mounting the rear wheels (only for quick release 22", 24" rear wheels).....	7
2.3 Mounting or removing of the footrests .....	7
2.4 Angle of footrests .....	8
2.5 Fold away leg support .....	8
2.6 Operating the brakes.....	8
2.7 Mounting or removing of arm supports .....	9
2.8 Pushing handles.....	10
2.9 Mounting or removing backrest .....	10
2.10 Transfer in and out the wheelchair .....	10
2.11 Correct position in the wheelchair .....	11
2.12 Riding the wheelchair (only for 22" and 24" rear wheels).....	11
2.13 Riding the wheelchair by an attendant .....	11
2.14 Moving on slopes .....	11
2.15 Negotiating steps or curbs.....	12
2.16 Taking off the wheels (only for quick release 22", 24" rear wheels).....	13
2.17 Backrest adjusting / Seat inclination.....	13
2.18 Transport in the car.....	14
2.19 Use of the wheelchair as seat in a motor vehicle .....	14
<b>3 Installation and adjustment</b> .....	<b>16</b>
3.1 Tools .....	17
3.2 Manner of delivery .....	17
3.3 Mounting backrest.....	17
3.4 Adjusting the seat height and seat angle .....	17
3.5 Adjust the seat depth .....	19
3.6 Adjust the backrest.....	20
3.7 Adjust the brakes .....	20
3.8 Adjusting of the footrests .....	21
3.9 Adjusting the armrests.....	23
3.10 Adjusting of the tip cap .....	24
<b>4 Maintenance</b> .....	<b>24</b>
4.1 Regular Maintenance.....	24
4.2 Shipping and storage .....	25
4.3 Care .....	25
4.4 Inspection .....	26
4.5 Disinfection .....	26
<b>5 Guarantee</b> .....	<b>28</b>
<b>6 Disposal</b> .....	<b>28</b>
<b>7 Declaration of conformity</b> .....	<b>28</b>
<b>8 Maintenance plan</b> .....	<b>29</b>
<b>9 Disinfection book</b> .....	<b>29</b>

## Preface

First of all we want to thank you for putting your trust in us by selecting one of our wheelchairs.

The Vermeiren wheelchairs are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the wheelchair.

The expected lifetime of your wheelchair is strongly influenced by the care and maintenance of the wheelchair.

This manual will help you get acquainted with the operation of your wheelchair.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

# 1 Productdescription

## 1.1 *Intended Use*

The wheelchair is intended for people with walking difficulties or no walking abilities.

The wheelchair is designed to transport 1 person.

The wheelchair is suited for indoor and limited outdoor use.

The user can propel the wheelchair by himself (only for 22", 24" rear wheels) or have the wheelchair pushed by an attendant.

The different types of fittings and accessories, and the modular construction allow full use by persons disabled by:

- paralysis
- loss of limbs (leg amputation)
- limb defects or deformations
- stiff or damaged joints
- heart insufficiencies and poor blood circulation
- balance disturbances
- cachexia (decrease in muscle)
- and also for aged persons.

When providing for individual requirements:

- body size and weight (max. 135 kg)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

Your wheelchair should only be used on surfaces where all four wheels are touching the ground and where there is sufficient contact to propel the wheels equally.

Ride only over flat surfaces with firm ground. It should not be used to drive on ground such as gravel, mud, cobble stones, grassy surfaces, or other ground that is not firm or level.

The wheelchair should not be used as a ladder, nor is it a transport for heavy or hot objects.

When used on mats, carpeted floors or loose floor coverings, the floor covering can get damaged.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.

## 1.2 Technical specifications

Technical terms below are valid for the wheelchair in standard settings.

If other footrests / arm supports or other accessories are used, the tabulated values will change.

<b>Make</b>	Vermeiren		
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout		
<b>Type</b>	Manual wheelchair		
<b>Model</b>	Inovys II		
<b>Maximum occupant mass</b>	135 kg		
<b>Description</b>	<b>Dimensions</b>		
Effective seat width	400 mm	450 mm	500 mm
Overall width (depends on the seat width)	630 mm	680 mm	730 mm
Dismantled width (without arm support)	600 mm	650 mm	700 mm
<b>Description</b>	<b>Dimensions Min.</b>	<b>Dimensions Max.</b>	
Overall length with footrests (depends on the adjusted seat depth)	1010 mm	1050 mm	
Dismantled length without footrests (depends on the adjusted seat depth)	830 mm	870 mm	
Dismantled height (Backrest folded, without arm support)	710 mm	780 mm	
Total mass	± 36 kg (24" rear wheels)	± 36,40 kg (16" rear wheels, drum brakes)	
Mass of heaviest part	± 20,70 kg (24" rear wheels)	± 25,20 kg (16" rear wheels, drum brakes)	
Masses of parts that can be dismantled or removed	<b>Footrests:</b> 4,25 kg; <b>Arm supports:</b> 3,45 kg; <b>Rear wheels 24":</b> 4,15 kg; <b>Backrest:</b> 3,45 kg		
Static stability downhill	10° (in standard configuration)		
Static stability uphill	15° (in standard configuration)		
Static stability sideways	12° (in standard configuration)		
Obstacle climbing	60 mm		
Seat plane angle	0°	+21°	
Effective seat depth	460 mm	520 mm	
Seat surface height at front edge (with cushion)	460 mm	530 mm	
Backrest angle	0°	+54°	
Backrest height	550 mm	585 mm	
Distance between footrest and seat	400 mm	570 mm (Do not use maximum distance with the footrest in vertical position -- touch the ground)	
Angle footplate	80°	100°	
Angle between seat and footrest	100°	190°	
Distance between armrest and seat	220 mm	320 mm	
Front location of armrest structure	460 mm	490 mm	
Hand rim diameter (only for 24" rear wheels)	535 mm		
Horizontal location of axle (deflection)	-8 mm (16" rear wheels) -4 mm (24" rear wheels)	41 mm (16" rear wheels) 42 mm (24" rear wheels)	
Minimum turning radius	1520 mm	1620 mm	
Diameter Krypton PU Rear wheels	16" (T30) + drum brakes	22"	24"
Diameter Krypton PU steering wheels	200 mm		
Color of frame	C55		

Cover	Soft grey	
Storage and use temperature	+ 5 °C	+ 41 °C
Storage and use humidity	30%	70%

**We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / °**

*Table 1: Technical specifications Inovys II*


The wheelchair complies to the requirements set up in:

ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

ISO 7176-16: Resistance to ignition of upholstered parts

ISO 7176-19: Use as seats in motor vehicles

### 1.3 Drawing

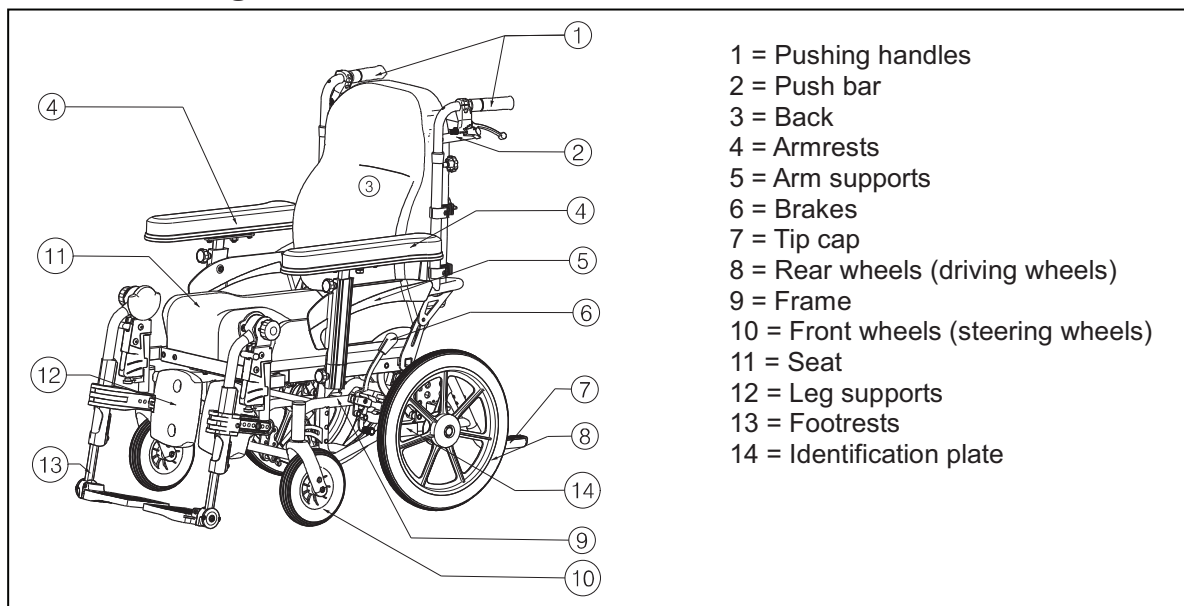### 1.4 Accessories

The following accessories are available for the Inovys II:

- Wooden table (B12)
- Headrest (L55)
- Headrest (L58)
- Personal safety belt (B58)
- Horizontal wedge (B22)
- Hemiplegic armrests (B66)

Contact your specialist dealer about other accessories. He will gladly advise you.

### 1.5 Explanation of symbols

- Maximum mass
- Indoor and outdoor use
- Down slope
- Up slope
- CE conformity

## 1.6 Safety rules

- ⚠ To prevent injury and/or damage to your wheelchair, make sure that no objects and/or body parts are caught in the spokes of the driving wheels (for 22", 24" rear wheels).
- ⚠ The parking brakes should be applied before getting into and out of the wheelchair.
- ⚠ When getting into and out of the wheelchair, do not stand on the footplates. These should be folded up beforehand, or the leg supports should be swung completely outwards out of the way.
- ⚠ Investigate the effects of shifting the centre of gravity on the behavior of the wheelchair, for example on up or down gradients, on laterally sloping ground, or when overcoming obstacles. Obtain support from an attendant.
- ⚠ If you want to pick up something (lying in front of, on the side, or to the rear of the wheelchair), you should not lean too far out to avoid tipping over.
- ⚠ When moving through doors, arches, etc. ensure that there is enough room at the sides so that you do not get your hands or arms caught or crushed and that there is no damage to the wheelchair.
- ⚠ Only use your wheelchair according to regulations. For example, avoid uncontrolled rolling against obstacles (steps, curbs, doorframes, etc) or dropping down from ledges. The manufacturer cannot assume liability for damage caused by overloading, collision or other improper use.
- ⚠ Stairs may only be negotiated when aided by another person. If furnishings like drive-on, ramps or lifts are available, use them.
- ⚠ When moving on the public highway, you are subject to the rules of the road.
- ⚠ When driving your wheelchair, you should not be under the influence of alcohol or medicine as in the case of driving other vehicles. This also applies to indoor driving.
- ⚠ When travelling outdoors, adapt your driving to weather and traffic conditions.
- ⚠ When transporting the wheelchair, never pick it up by grasping removable parts (arm supports, footrests, etc.)
- ⚠ To be better visible when driving in the dark, wear the brightest possible clothing or clothes with reflectors, and check that the reflectors mounted on the sides and rear of the wheelchair are clearly visible.
- ⚠ Be careful when using possible causes of fire such as cigarettes since they may set the seat and back covers alight.
- ⚠ Never exceed the maximum load of 135 kg.
- ⚠ Take care during all adjusting functions that no objects or body parts come into the area of the adjustment as there is the risk of squashing.
- ⚠ Check that the profile depth of the tyres is adequate.
- ⚠ To avoid hand injury, do not push your hand between the driving wheel and the knee lever brake when propelling your wheelchair (only for 22", 24" rear wheels with hand rim).
- ⚠ When driving outdoors, wear gloves which improve your grip on the tyre grips and protect your fingers against dirt and heating (only for 22", 24" rear wheels with hand rim).
- ⚠ Take care when attaching the quick-assembly wheels that the axle is completely and properly engaged (only for 22", 24" rear wheels with hand rim).

## 2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**


The wheelchair is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the wheelchair are given in § 3.

### 2.1 Carrying the wheelchair

The best way to carry the wheelchair is to make use of the wheels and roll the wheelchair.

If this is not possible (e.g. when the rear wheels are taken off for transportation in a car), firmly grasp the frame on the front and the back. Do not use the foot or arm rests or the wheels to grasp the wheelchair. At least two persons are required for transporting the wheelchair.


### 2.2 Mounting the rear wheels (only for quick release 22", 24" rear wheels)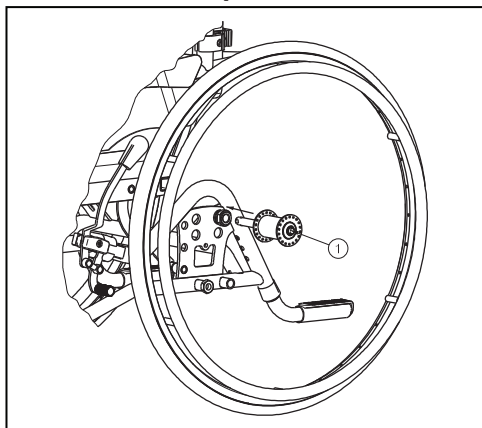

1. Take the rear wheel and push on button ①.
2. Keep the button pushed in and mount the rear wheels axle till it stops.
3. Release the button.
4. Check that the wheel is secured.

### 2.3 Mounting or removing of the footrests

**⚠ WARNING: Risk of injury - Make sure that the footrests are firmly locked before using the footrests.**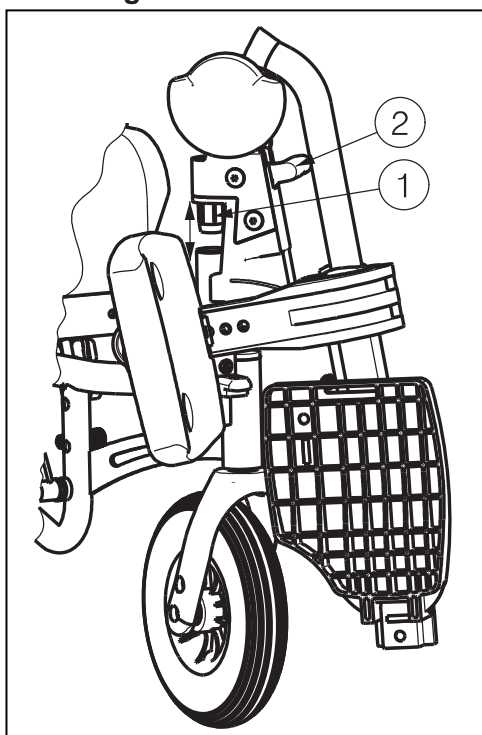

The mounting of the footrests is done as follows:


1. Hold the footrest sideways at the outside of the wheelchair frame and mount the tube hood ① into the frame.
2. Swing the footrest inwards till it clicks in position.
3. Swing the foot plate downwards.

To take off the footrests:

1. Pull lever ② upwards.
2. Swing the footrest to the outside of the wheelchair till it comes loose from the guidance.
3. Pull the footrest from tube hood ①.

## 2.4 Angle of footrests


- ⚠ **CAUTION:** Risk of injury – Check that the adjusting button ① is well fixated after adjusting the angle of the footrest.
- ⚠ **CAUTION:** Risk of clamping – Keep fingers, buckles and clothes away from the adjusting mechanism. To adjust the angle of the footrest, only touch them at the level of the footplates-to avoid injury on your fingers.
- ⚠ **CAUTION:** Risk of injury – Make sure that no objects and/or persons are inside the swinging range of the adjust functions.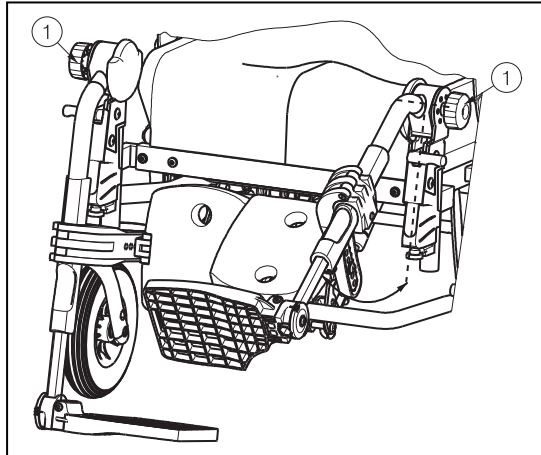

The angle or inclination of the footrest can be easily adjusted as follow:

1. Loosen the adjusting button ① on the side of the footrest.
2. Adjust the angle (range 100° - 190° step less) of the footrest to a comfortable position. Grasp the footrest at the level of the footplates.
3. Retighten the adjusting button ① firmly by hand.

## 2.5 Fold away leg support

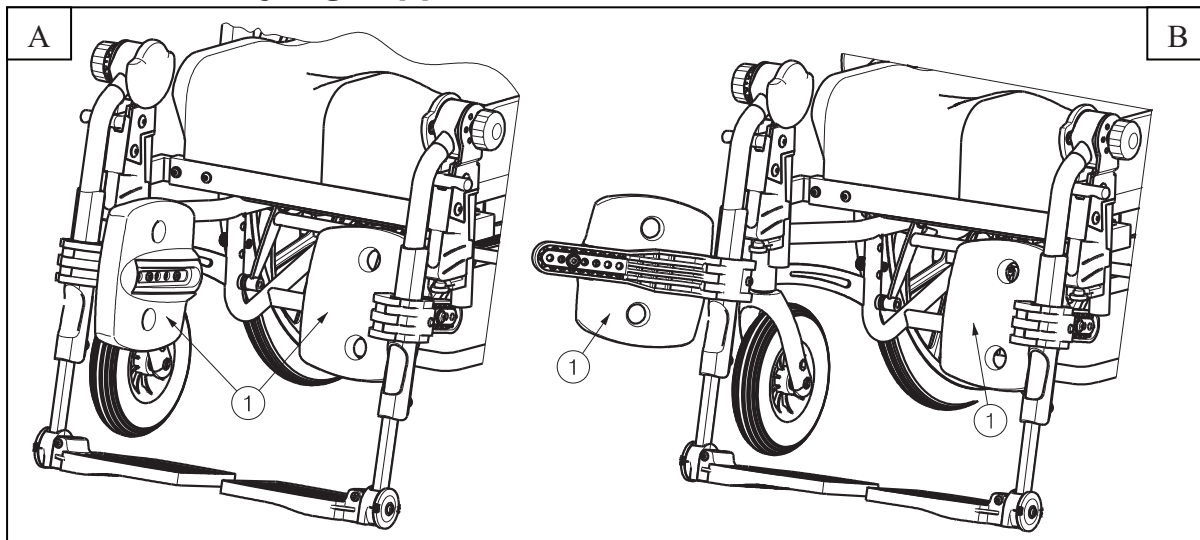The leg support can be easily fold away as follow:


1. Take the leg support ① on the outside. (Figure A)
2. Fold the leg support ① to the front. (Figure A)
3. Take the leg support ① on the back. (Figure B)
4. Swing the leg support ① forwards. (Figure B)

To put the leg supports in the begin position you can do the reverse. **Check you do not place your fingers between the movable parts or were they can be squeezed.**

## 2.6 Operating the brakes

- ⚠ **WARNING:** The brakes are not used to slow down the wheelchair during movements – Use the brake only to prevent the wheelchair from unintended movements.
- ⚠ **WARNING:** Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) – Check the condition of the tires before each use.

**⚠ WARNING: The brakes are adjustable and can wear – Check the operation of the brakes before each use.**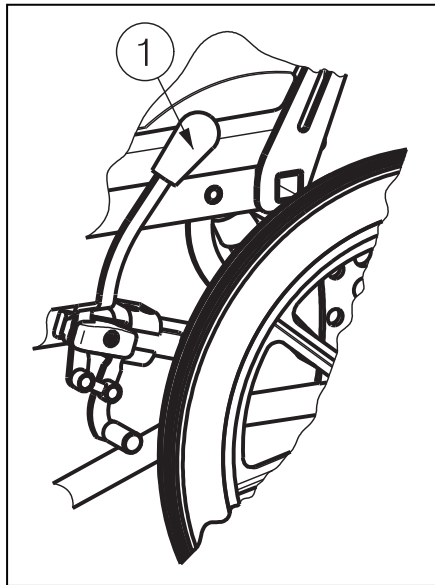

To apply the brakes:

1. Push the brake handles ① forward till you feel a distinctive click.


**⚠ CAUTION: Risk of unintended movement – Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.**

To release the brakes:

1. Release one brake by pulling the handle ① backwards.
2. Hold the hand rim (if available) of the release wheel with your hand.
3. Release the second brake by pulling the handle ① backwards.

## 2.7 Mounting or removing of arm supports

**⚠ CAUTION: Risk of clamping – Keep fingers, buckles and clothes away from the bottom side of the arm support.**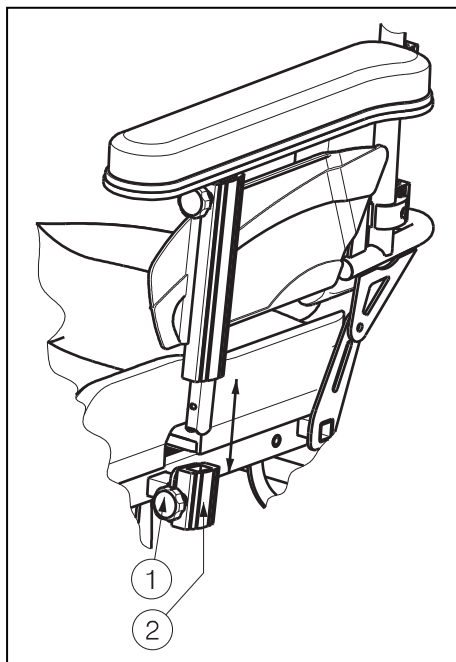


The arm supports of the wheelchair can be mounted by following instructions.

1. Pull out the star knob ①.
2. Mount the arm support from above into the provided bracket ②.
3. Let go of the star knob ①.
4. Turn the star knob ① clockwise until it is finger-tight.
5. Check that the arm support is firmly secured.

The arm supports of the wheelchair can be removed by following instructions.

1. Turn the star knob ① anti-clockwise.
2. Pull out the star knob ①.
3. Remove the arm support out the bracket ②.
4. Let go of the star knob ①.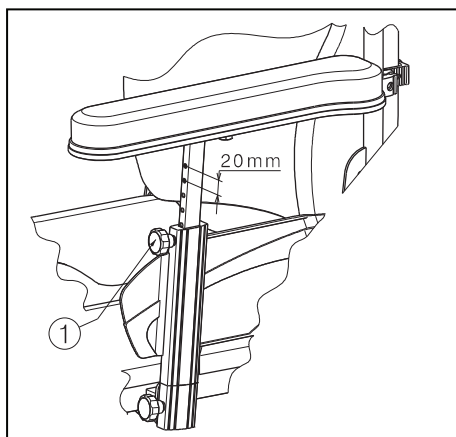


The armrest of the wheelchair can be adjusted in 6 different heights (steps from 20 mm):

1. Turn the star knob ① anti-clockwise.
2. Pull out the star knob ①.
3. Pull the armrest into the required position.
4. Let go of the star knob ①.
5. Turn the star knob ① clockwise until it is finger-tight.
6. Check that the armrest is firmly secured.

## 2.8 Pushing handles

- ⚠ **CAUTION:** Risk of injury - Make sure that the star buttons ① are properly tightened.
- ⚠ **CAUTION:** Risk of tipping over - Use only the last indentation of the pushing handle tube for the maximum height adjustment.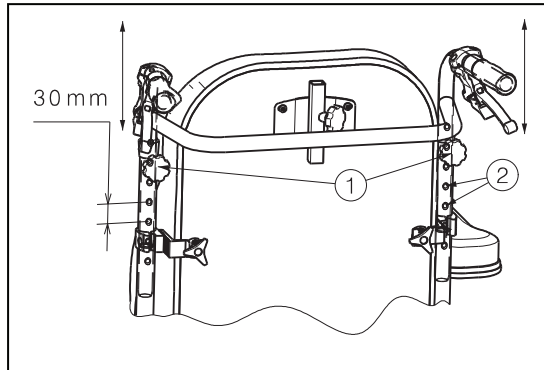


The pushing handles can be adjusted in height depending on the size of the attendant.

1. Loosen the star button ① located on the back of the backrest.
2. Pull the pushing handles into the desired position (7 positions). Indentations ② (steps 30 mm) have been placed into the tubing of the pushing handles.
3. Retighten the star button ① firmly by hand.

## 2.9 Mounting or removing backrest

- ⚠ **WARNING:** Risk of injury – Check that the backrest is firmly mounted into the four brackets before using your wheelchair.
- ⚠ **WARNING:** Risk of injury – Avoid getting your fingers, clothes caught during the assembly process.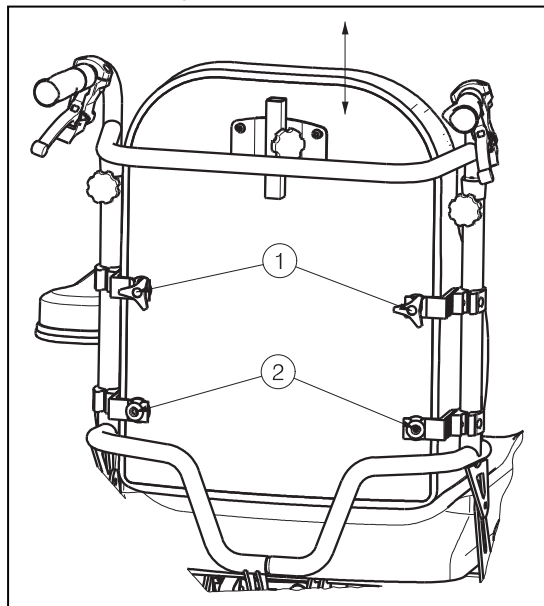

The backrest of the wheelchair can be mounted by following instructions.


1. Mount the backrest into the four brackets ① + ②.
2. Turn on the upper star buttons ① to lock the star buttons by hand.
3. Check that the backrest is firmly fixated.

To remove the backrest of the wheelchair:

1. Loosen the upper star buttons ①.
2. Pull the backrest upwards out the four brackets ① + ②.

## 2.10 Transfer in and out the wheelchair

- ⚠ **CAUTION:** In case you cannot perform the transfer in a safe manner, ask someone to assist you.
- ⚠ **CAUTION:** Risk of tipping over of the wheelchair – Do not stand on the foot plates.

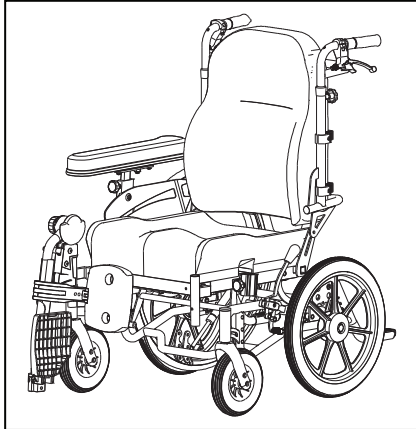1. Place the backrest in the most vertical position.
2. Position the wheelchair as close as possible to the chair, couch or bed to/from you wish to transfer.
3. Check both brakes from the wheelchair are in the on position.
4. Fold the foot plates upwards to prevent standing on them.
5. If the transfer is on the side of the wheelchair, you can remove the arm support (see § 2.7) and/or footrest (see § 2.3) on that side.
6. Transfer to/from the wheelchair.

## 2.11 Correct position in the wheelchair

Some recommendations for a comfortable use of the wheelchair:

- Position your backside as close as possible to the back rest.
- Make sure your upper legs are horizontal – If needed adjust the length of the footrests. (see § 3.8.1)

## 2.12 Riding the wheelchair (only for 22" and 24" rear wheels)

- ⚠ **WARNING:** Risk of clamping – Prevent your fingers from being caught by the wheels spokes.
- ⚠ **WARNING:** Risk of clamping – Be careful passing through restricted passages (e.g. doors).
- ⚠ **WARNING:** Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.

1. Release the brakes.
2. Take both hand rims at their highest position.
3. Lean forward and push the hand rims forward until straight arms.
4. Swing your arms loosely back to the hand rims topside and repeat the movement.

## 2.13 Riding the wheelchair by an attendant

- ⚠ **WARNING:** Risk of clamping – Be careful passing through restricted passages (e.g. doors).
- ⚠ **WARNING:** Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.

The wheelchair can be pushed with the pushing handles or push bar by an attendant.

## 2.14 Moving on slopes

- ⚠ **WARNING:** Control your speed – Moving on slopes as slow as possible.
- ⚠ **WARNING:** Consider the capacities of your attendant – If your attendant does not have enough force to control the wheelchair, put on the brakes.
- ⚠ **WARNING:** Risk of tipping over – Lean forward to move your centre of gravity forward. To improve a better stability.

1. If available on the wheelchair, wear the safety belt.
2. Do not attempt moving on too high slopes. The maximum slope angles (upwards and downwards) are mentioned in table 1.
3. Ask an attendant to help you moving on the slope.
4. Lean forward to move your centre of gravity forward.

## 2.15 Negotiating steps or curbs

- ⚠ **WARNING:** Risk of injury – Transporting a patient up or down steps is to be done with at least two attendants.
- ⚠ **WARNING:** Risk of injury – Wear suitable safety belt systems while being transported (information obtainable from your specialist dealer).
- ⚠ **WARNING:** Risk of injury – Sit quietly in your wheelchair and avoid jerky movements. If you cannot sit quietly when being transported because of illness or disability (e.g. slightly spastic, etc.), then at least two persons are required to secure the wheelchair on both sides by taking hold of fixed parts of the frame.
- ⚠ **WARNING:** Risk of injury – Make sure that your arms and legs do not protrude outside the wheelchair while being transported.
- ⚠ **WARNING:** Risk of injury or damages – No persons or objects should be below the wheelchair while being transported.
- ⚠ **WARNING:** Risk of injury – When the footrests have been removed, the legs of the patient are not secured.

### 2.15.1 Getting down steps or curbs

Riding down steps or curbs can be done moving forwards with an attendant:

1. Make sure that the pushing handles are sufficiently tight.
2. Remove the footrests.
3. Ask the attendant to tip the wheelchair slightly backwards with the tip cap.
4. Get past the curbs while moving on the rear wheels and push the wheelchair forwards.
5. The second person secure the wheelchair in front by taking hold of the front of the frame after removing the footrests. He must walk backwards.
6. Put the wheelchair back on the four wheels after passing the curbs.

### 2.15.2 Moving up steps or curbs

Moving up steps or curbs can be done moving backwards with an attendant:

1. Make sure that the pushing handles are sufficiently tight.
2. Remove the footrests.
3. Turn the wheelchair to have the rear wheels facing the curb.
4. Lean backwards and move your centre of gravity above the rear wheels.
5. Ask the attendant to pull the wheelchair on the curb.
6. The second person secure the wheelchair in front by taking hold of the front of the frame after removing the footrests. He must walk forwards.
7. Take back the normal position in the wheelchair.

### 2.15.3 Taking of stairs

- ⚠ **WARNING:** Risk of tipping over – Taking of stairs shall always be done with 2 attendants.
- ⚠ **WARNING:** Risk of injury – Lifting the wheelchair while the patient is seated in the wheelchair is forbidden! Always keep the rear wheels on the ground/stairs.
- ⚠ **WARNING:** Risk of injury – Wear suitable safety belt systems while being transported (information obtainable from your specialist dealer).
- ⚠ **WARNING:** Risk of injury or damages – No persons or objects should be below the wheelchair while being transported.
- ⚠ **WARNING:** Risk of injury – When the footrests have been removed, the legs of the patient are not secured.

Taking the wheelchair of stairs shall be according following rules:

1. Remove the footrests.
2. One attendant tip the wheelchair slightly backwards.
3. The second attendant take the front of the frame.
4. Stay calm, avoid sudden movements and keep your arms inside the wheelchair.
5. Take the steps on the rear wheels of the wheelchair.
6. Mounting the footrests back after taking the stair.

#### 2.15.4 Using ramps

- ⚠ **WARNING: Risk of injury – Do not exceed the maximum load or the ramps.**
- ⚠ **WARNING: Risk of injury - Choose the correct ramps so there cannot be caused any injury or damage. We do not accept any liability for injury or damage caused by an improper choice of ramps.**
- ⚠ **WARNING: Risk of injury - Be sure that the wheel height is high enough to get over the free height of the ramps. The wheelchair frame may not touch the ramps.**
- ⚠ **WARNING: Risk of injury - Use a restraining safety belt to secure yourself in your wheelchair.**
- ⚠ **WARNING: Risk of tipping over - Put the adjustment functions (seat, back, footrests, ...) that the wheelchair has the best stability.**


If you are considering the installation of ramps to overcome obstacles, kindly note the following:

Because of the large weight of the wheelchair we advise against using your own power for driving on a ramp, since it might not be possible for you to prevent the wheelchair from rolling backwards.

**Never go up or down ramps without the aid of a helper.**

#### 2.16 Taking off the wheels (only for quick release 22", 24" rear wheels)

To facilitate the transport off the wheelchair the rear wheels can be taken off: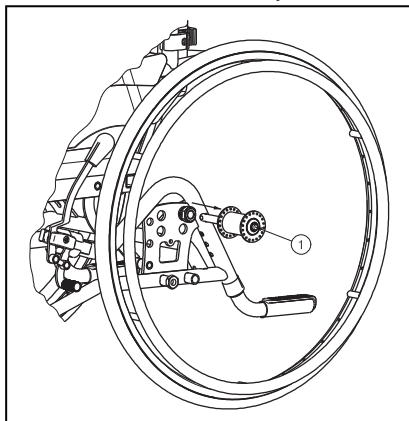


1. Make sure the brakes standing in the off position.
2. Take the wheelchair to the side frame where you want to remove the wheel.
3. Press the button ① in the center of the wheel hub.
4. Pull the wheel away from the frame.

#### 2.17 Backrest adjusting / Seat inclination

- ⚠ **WARNING: Risk of tipping over – Be aware that the stability decrease when the backrest is adjusted backwards.**
- ⚠ **WARNING: Risk of injury – Apply the parking brakes before adjusting the backrest and seat inclination.**

The seat and the backrest can only be adjusted simultaneously or independently from one another in their angle by an attendant. Be sure that the patient sits in the chair when the attendant shall perform the back adjusting, seat inclination and that the wheelchair does not tip over.

### Seat inclination:

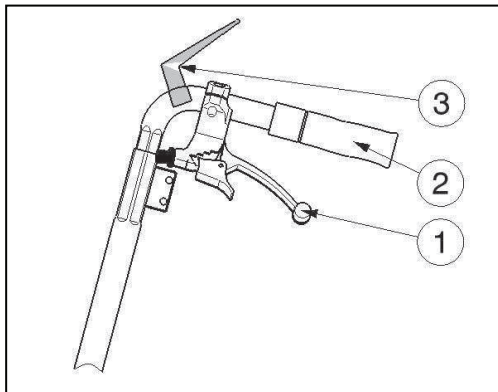1. Pull lever ① on the left side (see label) towards the grab handle ② to adjust the seat inclination.

Label

2. Pull/Push the seat in the desired position (0° to + 21°), the gas pressure spring shall be pressed in/out.
3. Loosen the lever ①, to block the gas pressure spring.

### Backrest adjusting:

1. Pull lever ① on the right side (see label) towards the grab handle ② to adjust the angle of the backrest.

Label

2. Pull/Push the backrest in the desired position (0° to + 54°), the gas pressure spring shall be pressed in/out.
3. Loosen the lever ①, to block the gas pressure spring.

**If your wheelchair features drum brakes, the levers ① are used as brakes; additional levers ③ have been mounted for adjusting angle of backrest and seat inclination. These levers ③ can be triggered with the thumb.**

## **2.18 Transport in the car**

- ⚠ WARNING: Risk of injury – See that the wheelchair is attached properly. So you can avoid injury from the passengers during collision or sudden braking.**
- ⚠ WARNING: Risk of injury – Use for attaching the wheelchair and passenger NEVER the same seatbelt.**

To transport the wheelchair in the car use following steps:

1. Set the seat in a horizontal position and the backrest vertically.
2. Remove footrests, arm supports and accessories (ex. headrest, ...).
3. Store footrests, arm supports and accessories safely.
4. If possible, fold the wheelchair and remove the wheels.
5. Place the wheelchair in the luggage place with 2 persons.
6. If the wheelchair and the passenger compartment is NOT separated, attach the frame of the wheelchair securely to the vehicle. You can use the available safety belts in the vehicle.
7. Place the patient in the recommended existing seats with their seatbelt systems.

## **2.19 Use of the wheelchair as seat in a motor vehicle**

- ⚠ WARNING: The wheelchair has passed the crash of ISO 7176-19: 2008 and, as such, has been designed and tested for use only as forward-facing seat in a motor vehicle.**
- ⚠ WARNING: The wheelchair's pelvic belt alone is not suited as an occupant restraint belt.**

The wheelchair is tested using the four-point strap-tie system and a 3-point occupant-restraint system.

Whenever feasible, use the seat of the vehicle and store the wheelchair in the cargo area.

Steps to secure the wheelchair in a motor vehicle:

1. Check that the vehicle is equipped with a suitable wheelchair tie down and occupant-restraint system, conform ISO 10542.
2. Check that the components of the wheelchair tie down and occupant restraint system are not frayed, contaminated, damaged or broken.
3. If equipped with an adjustable seat and/or back tilt, make sure that the wheelchair user is sitting as upright as possible. If the user's condition prevents this, a risk assessment should be done to evaluate the user's safety during transit.
4. Remove all mounted accessories such as trays and respiratory equipment, and secure them in a safe place.
5. Position the wheelchair facing forward in the travelling direction, centrally between the tie-down rails mounted in the floor of the vehicle.
6. Make sure that the indicated zones around the wheelchair user are clear from rigid vehicle parts.

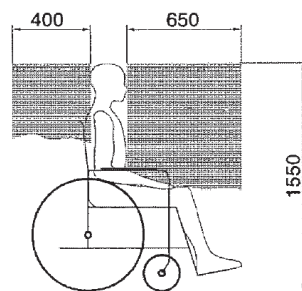
figure 1

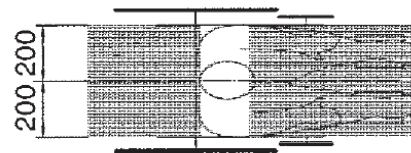
figure 2

7. Mount the front securement straps according to the instructions of the strap-system manufacturer at the indicated place. (figure 3)  
This place is marked on the wheelchair with a symbol. (figure 4)
8. Roll back the wheelchair until the front straps are tight.
9. Apply the wheelchair brakes.
10. Mount the back securement straps according to the instructions of the strap-system manufacturer at the indicated place. (figure 3)
11. This place is marked on the wheelchair with a symbol. (figure 4)


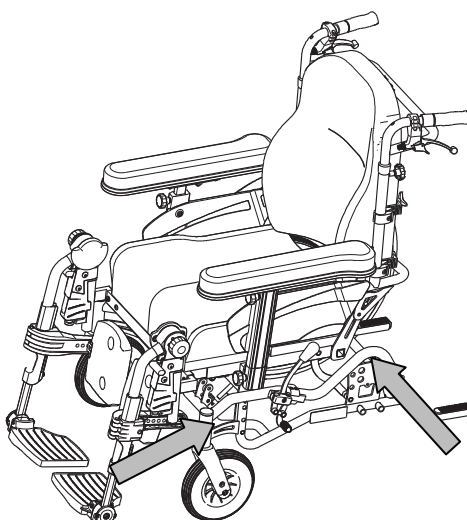
figure 3

figure 4

Steps to secure the wheelchair user:

1. Remove both arm rests.
2. If present, attach the wheelchair's pelvic belt.
3. Attach the occupant restraint belts according to the instructions of the strap-system manufacturer.
4. Wear the pelvic belt low across the front of the pelvis, so that the angle of the pelvic belt is within the preferred zone of 30° to 75° to the horizontal, similar to that shown below.

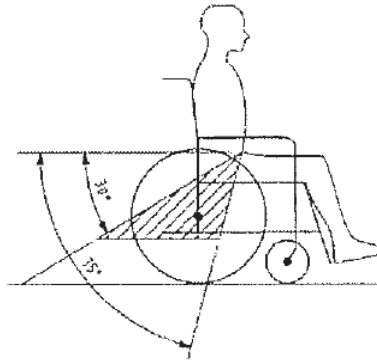5. A steeper (greater) angle within the preferred zone is desirable.
6. Adjust the belt tightly according to the instructions of the strap-system manufacturer, consistent with the user's comfort.
7. Ensure that the restraint belt connects in a straight line to the anchor point in the vehicle and that no bends in the belt are visible, for instance at the axle of the rear wheel.
8. Install the arm rests, if desired. Make sure that belts are not twisted or held away from the body by wheelchair components such as arm rests or wheels.

### 3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

The Vermeiren multi-functional wheelchairs Inovys II has been designed entirely for your comfort. This wheelchair can be adjusted to individual requirements as explained below.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

- ⚠ **WARNING:** Risk of unsafe settings - Use only the settings described in this manual.
- ⚠ **WARNING:** Variation of allowed adjustments can still change the stability of your wheelchair (tilt back or sideways).

### 3.1 Tools

To set up the wheelchair the following tools are needed.

- Wrench set n° 10 to n° 22
- Allen key set n° 3 to n° 8
- Screwdriver Phillips head
- Screwdriver
- Nipper
- Pliers

Assembly and settings may only be performed by a specialist dealer.

### 3.2 Manner of delivery


The Vermeiren Inovys II shall be delivered with:

- 1 frame with arm supports, rear and front wheels
- 1 pair footrests
- Seat and back cushions
- Tools
- Manual
- Accessories (optional)

When you receive the wheelchair, please check that the product supplied is complete and fully functional. Warranty claims are only valid if the manufacturer (supplier) is notified in writing within a period of two weeks. In the case of any defect that is not immediately evident, warranty claims are only valid if the manufacturer (supplier) is notified in writing within two weeks after it is detected. We shall remedy defects in the manner of our choosing - either by carrying out a repair or by providing a replacement.

### 3.3 Mounting backrest

**⚠ WARNING: Risk of injury - Make sure that the ring of safety pin **Ⓐ** is firmly secured.**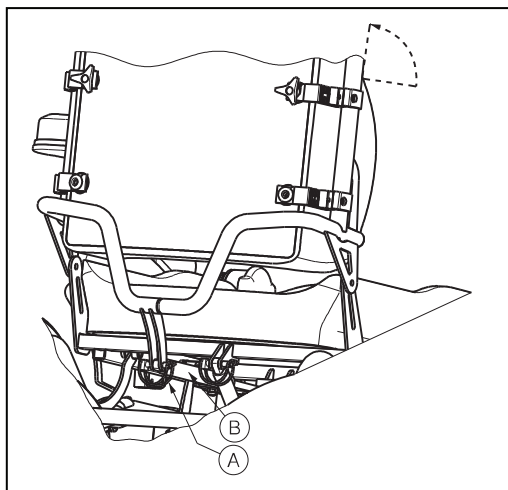

After you have taken the wheelchair out of the original packaging, push the backrest to the back into the 90° position. Guide the safety pins **Ⓐ** into the appropriate socket to attach the spring **Ⓑ**, which is located at the back under the seat.

Check that no cables are squashed.

### 3.4 Adjusting the seat height and seat angle


- ⚠ WARNING: Risk of injury - Make sure that all screws are properly tightened with the appropriate tool.**
- ⚠ WARNING: Risk of injury - Self-locking nuts are only to be used once or are to be secured with an additional screw locking device.**
- ⚠ WARNING: Risk of injury - Adjust the brakes if necessary after adjusting the seat height and seat angle.**

- ⚠ **WARNING: Risk of injury - Check that the seat heights are mounted in the same position on both sides.**
- ⚠ **WARNING: Risk of tipping over - When the driving wheels are adjusted, the position of the wheels are altered.**

The seat height of the Inovys II multiposition wheelchair is adjustable in 3 positions in steps of 35 mm. The seat height can be adjusted by moving the front tube (A), attachment plate gas pressure spring (B) and rear wheel plate (C) in different positions. These components you can find below the seat frame.

Seat height	Part (A, B, C)
460 mm	Hole 1
495 mm	Hole 2
530 mm	Hole 3

Table 2: Seat heights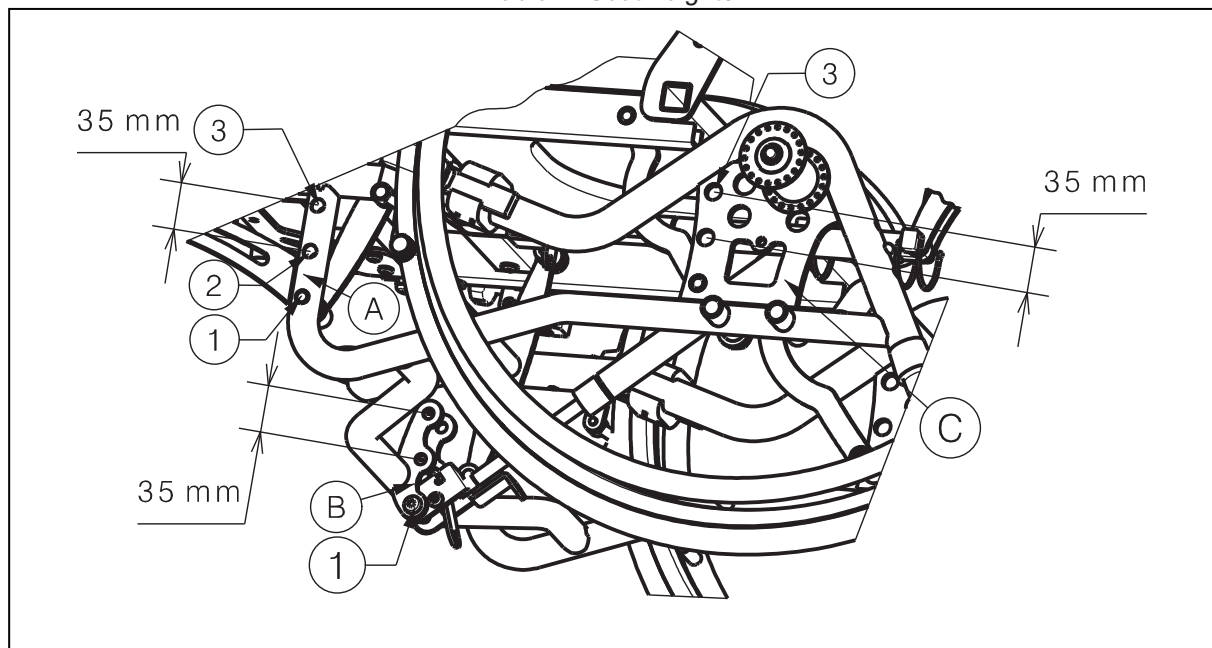

#### **(A) Front tube**

The basic frame and the seat frame are screwed together with the front tube (A). In the basic frame three different sockets are provided for these screws which determine the wheelchair's seat height.


#### **(B) Attachment plate gas pressure spring**

In addition to adjusting the frame, the gas pressure spring for the seat inclination can be fixed at three different attaching heights (B). So that the angle of inclination can be kept for the seat inclination, the assembly height of the gas pressure spring (B) must correspond to the assembly height of the front tube (A).

#### **(C) Rear wheel plate**

The rear wheel plate (C) must correspond to the assembly height of the front tube (A).

- ⚠ **WARNING: Risk of tipping over – Be aware that you change the stability of the wheelchair when changing the position of the driving wheels.**
- ⚠ **WARNING: Risk of injury – Make sure that the rear wheels on both sides are mounted in the same position.**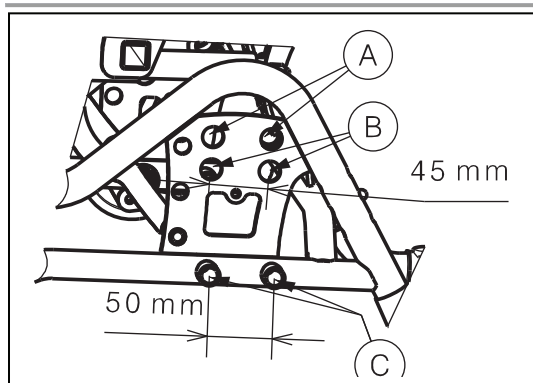

### DRIVING WHEELS

Four possible wheel sockets are provided in a swage block of the frame for the use of driving wheels (24") ①, (22") ②. Depending on which suspensions are used to attach the driving wheels, the wheel position (stability or manoeuvrability of wheelchair: 45 mm) and the seat height can be changed. The wheel sockets attached in the delivery condition can be attached in all sockets (bore holing)

The wheel sockets in the frame tubing ③ serve only to attach 16" wheels (transit version). The stability or manoeuvrability of the wheelchair can be adjusted over 50 mm.

### STEERING WHEELS

In the standard model, the INOVYS II is delivered with two 8" steering wheels (200 mm x 50 mm or 200 mm x 35 mm).


## 3.5 Adjust the seat depth

**⚠ WARNING: Risk of injury – Make sure that all screws are secure and properly tightened.**

The seat depth can be adjusted in 3 positions (steps 20 mm). Adjust the seat depth according following rules:

Seat depth	Hole
480 mm	①
500 mm	②
520 mm	③


Table 3: Seat depth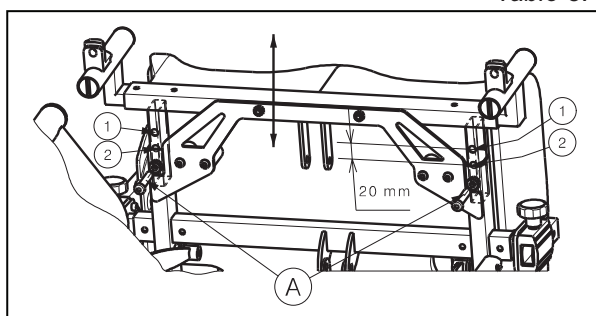

1. Loosen the screws ① beneath the front seat frame.
2. Pull or push the front of the seat frame into the desired position.
3. Retighten the screws ① firmly again.

**Note:** When rebuilding the seat depth in the back frame area, the displacement of the spring for adjusting the back is changed and a gap can occur between the back and seat cushions.


The seat depth can also be changed by adjusting the brackets of the backrest. With these adjustment you can become a minimum seat depth of 460 mm.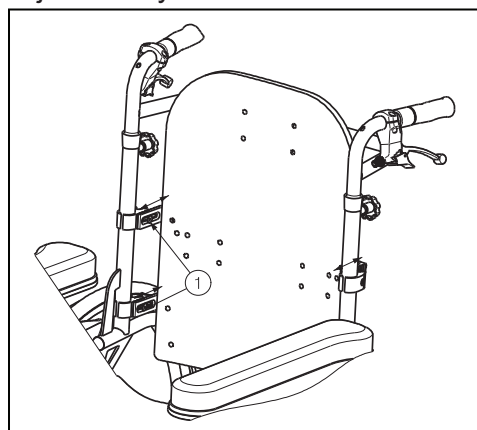

1. Remove the backrest.
2. Loosen the bolts ① of the four brackets.
3. Adjust the brackets more to the front or back until the desired seat depth.
4. Retighten the bolts ① of the four brackets.
5. Replace the backrest.

### 3.6 Adjust the backrest


The height of the backrest can be adjusted over a distance of 35 mm (step less) according following instructions: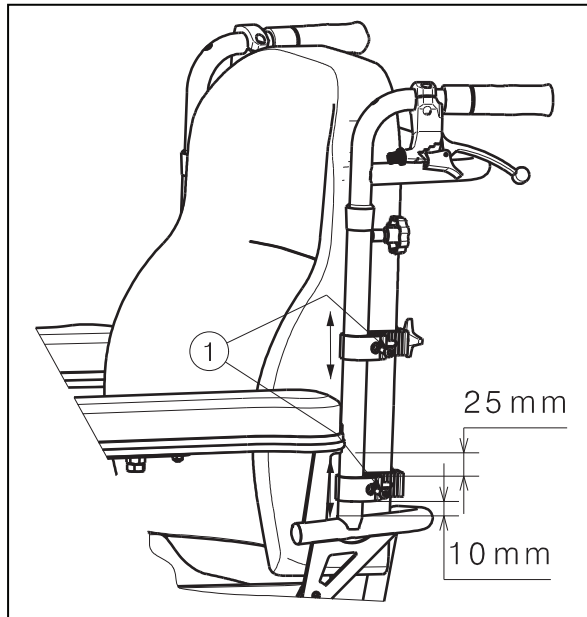


1. Loosen the bolts ① of the four brackets.
2. Adjust the brackets of the backrest to the desired height.
3. Retighten the bolts ① of the four brackets.

### 3.7 Adjust the brakes

Adjust the brakes according following rules: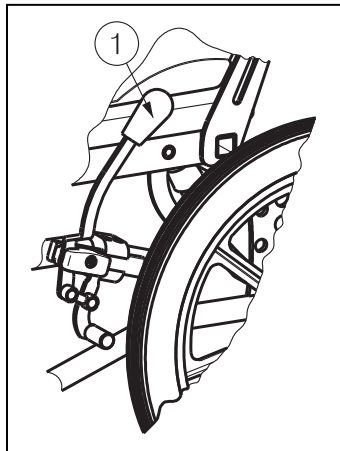

1. Install the wheels according § 2.2 for quick release 22", 24" rear wheels.
2. Disconnect the brakes by pulling lever ① backwards.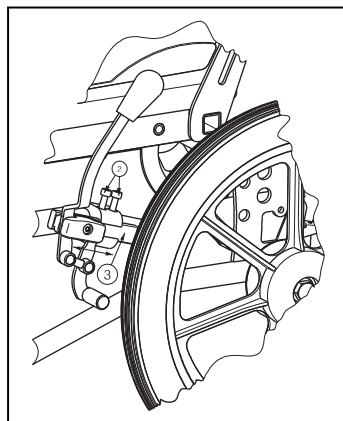


3. Loosen the bolts ② so the brake mechanism can slide over the guide ③.
4. Pull the brake mechanism over the guide ③ to the desired position.
5. Retighten the bolts ②.
6. Check working of the brakes.
7. If necessary repeat the above steps until the brakes are adjusted well.

## 3.8 Adjusting of the footrests


### 3.8.1 Length of the footrests

**⚠ CAUTION:** Risk of damage – Avoid that the footrests make any contact with the ground. Keep a minimum distance from 60 mm above the ground.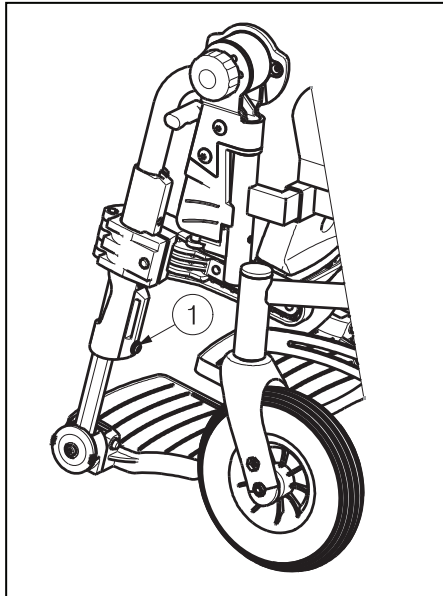

Adjust the length of the footrests as follow:

1. Remove the screw ① (on the back of the footrests).
2. Adjust the length of the footrest to a comfortable length.
3. Tighten the screw ① properly.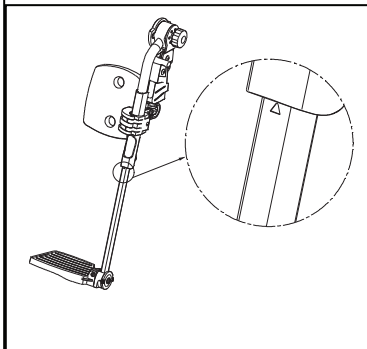


**The max adjustable length of the footrest shall be marked with a triangle on the tube.**

**Not possible when footrest is in vertical position.**

### 3.8.2 Adjusting leg support

The leg supports are available in different sizes. Small size for seat width 400 mm wheelchair and medium size for wheelchair with seat widths 450 mm and 500 mm.


**Adjusting the height:**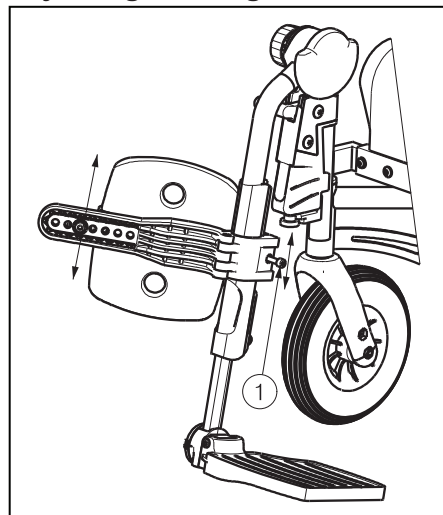


1. Fold away leg support (according paragraph 2.5).
2. Loosen bolt ① on the side of the footrest.
3. Adjust the height of the leg support to a comfortable position (step less).
4. Retighten the bolt ①.

### Adjusting the depth:

1. Loosen bolt ① on plate ④.
2. Adjust the depth of the leg support to a comfortable position (7 different positions in steps of 15 mm).
3. Retighten the bolt ①.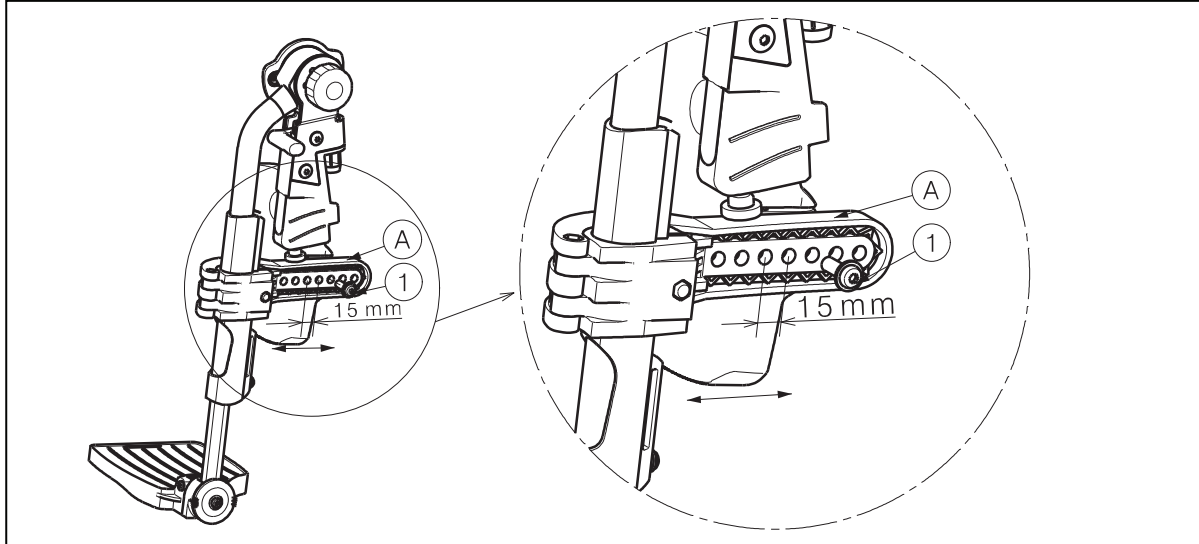


### Adjusting the width:

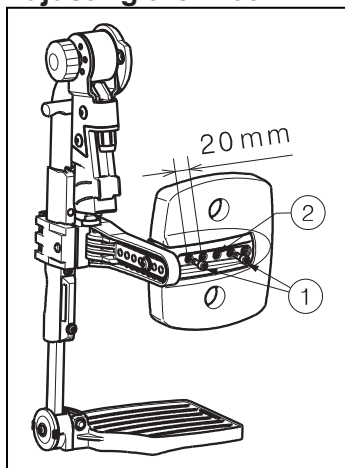1. Loosen the two bolts ① on the back of leg support cushion.
2. Adjust the width of the leg support to a comfortable position (2 different positions in steps of 20 mm, **do not use the middle hole ② for adjusting**).
3. Retighten the two bolts ①.

### 3.8.3 Adjusting angle foot plate

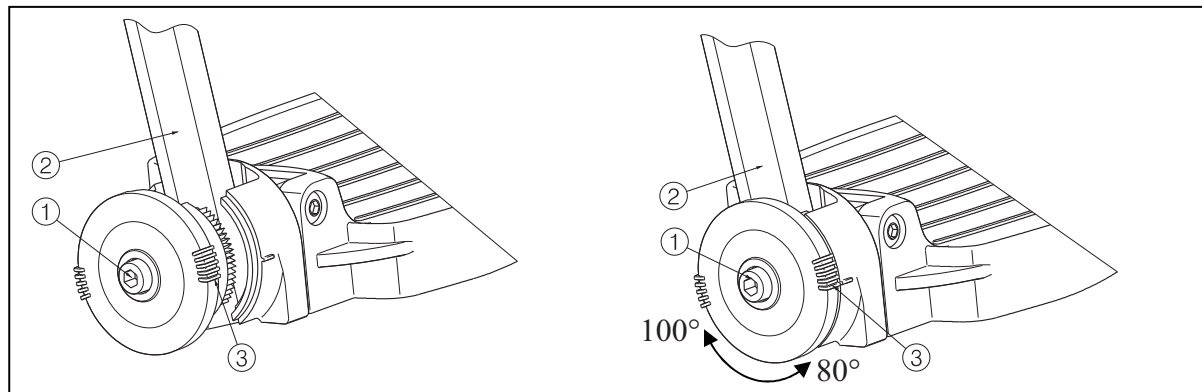
Adjust the angle of the footplate as follow:

1. Loosen bolt ①.
2. Adjust tube ② in the desired angle. The angle indication (80°-85°-90°-95°-100°) is according the dashes ③.
3. Retighten the bolt ①.

### 3.8.4 Adjusting the width of footrest

- ⚠ **WARNING:** Risk of injury – Make sure that all screws are secure and properly tightened before using the footrests.
- ⚠ **WARNING:** Risk of injury – Make sure that the footrests are positioned symmetrically from the seat.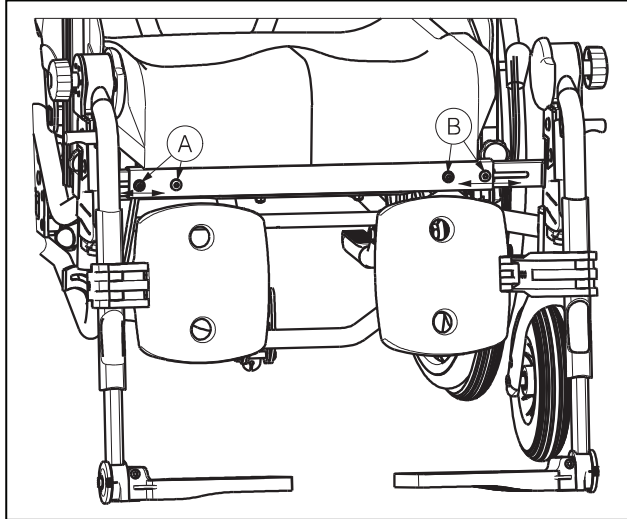


Adjust the width of the footrest as follow:

1. Remove the bolts Ⓐ for right footrest and Ⓑ for left footrest on the front of the seat frame.
2. Adjust the footrest in the desired position.
3. Retighten the bolts Ⓐ and Ⓑ properly by hand.

**Pull out the footrest socket only to the extent that both screws on one side still secure the footrest socket.**

## 3.9 Adjusting the armrests


### 3.9.1 Depth adjusting of armrests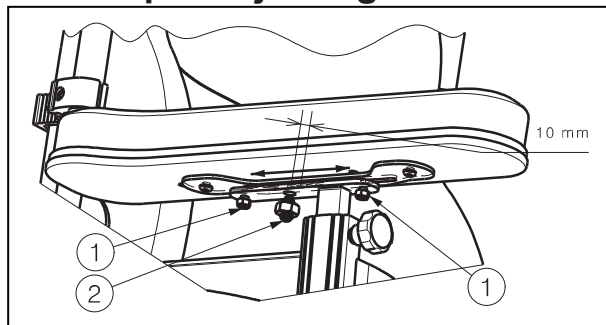

1. Loosen the fixing screws ① beneath the arm cushion.
2. Remove the nut + spring plunger ②.
3. Push the arm cushion into the desired position.
4. Retighten the fixing screws ① and nut + spring plunger ②.

### 3.9.2 Width adjusting of the armrests

- ⚠ **WARNING:** Risk of injury – Make sure that all screws are secure and properly tightened before using the armrests.
- ⚠ **WARNING:** Risk of injury – Make sure that the armrests are positioned symmetrically from the seat.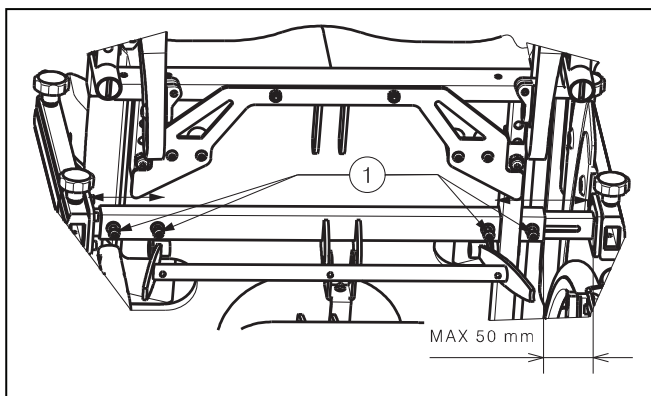

1. Loosen the screws ① of the armrest sockets beneath the seat.
2. Pull the armrest socket into the desired position (max. of 50 mm outwards).
3. Retighten the screws ① firmly by hand.


**Pull out the armrest socket only to the extent that both screws on one side still secure the armrest socket.**

### 3.10 Adjusting of the tip cap

**⚠ WARNING: Risk of injury - Make sure that the spring button ① is locked after adjusting the tip cap.**

A tip cap is mounted on the lower back frame that can be adjusted in its height depending on the seat height of the wheelchair.

The tip cap can be placed in 3 different positions with a difference of 20 mm.

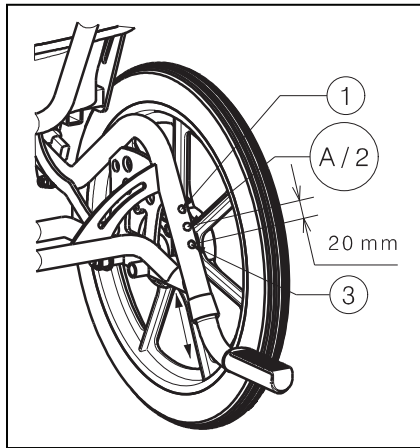1. Press the spring button ①.
2. Press the tip cap tube into the frame tube or out the frame tube until the desired position is reached.
3. Loosen the spring button ①.
4. Check that the spring button ① is securely fixated.

## 4 Maintenance

Lifetime of the wheelchair is influenced by its use, storage, regular maintenance, servicing and cleaning.

### 4.1 Regular Maintenance

The maintenance activities to ensure a good condition for your Vermeiren wheelchair are described below:

- Before every ride
  - Check the tyres and make them clean. If necessary change the tire.
  - Check screw fastenings, levers, adjusting buttons are secured.
  - Check the brakes and adjusted if necessary.
  - Check the wheelchair (cleanliness, cracks, damage on structural parts ... .) and make it clean. If necessary change the coating.
- Every 8 weeks: inspected and lubricating or adjusting from
  - Arm supports
  - Footrests
  - Brake lever
  - Wheel axles
  - Adjusting systems
- Every 6 months or for every new user
  - General review
  - Disinfection
  - Frontwheels, possibly clean bearings

For your convenience, on the back of this manual is a maintenance plan.

Repairing and assembling of spare parts for your wheelchair may only be performed by the specialist dealer.

Only authorized Vermeiren spare parts may be installed.

## 4.2 Shipping and storage

The shipping and storage of the wheelchair shall be according following instructions:

- Store in a dry place (between +5 °C and +41 °C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the wheelchair from rust and foreign bodies. (e.g. salt water, sea air, sand, dust).
- Store all removed parts together in one place (or mark them if necessary) to avoid mixing up with other products when re-assembling.
- Components must be stored without being subjected to strains (Do not put too heavy parts on the wheelchair, not clamping between something, ...).

## 4.3 Care

### 4.3.1 Seat and back cushion

The cleaning of the seat and back cushion shall be according:

- Clean seat and back with a cloth moistened with warm water. Be aware that you do not soak the seat and back cushion.
- Use a mild commercial detergent for removing stubborn dirt.
- Stains can be removed by using a sponge or a soft brush.
- Do not use strong cleaning liquids like solvents, nor use hard brushes, sharp-edged objects.
- Never clean with steam and/or pressure washers.

### 4.3.2 Plastic parts

Clean plastic parts of your wheelchair with commercial plastic cleaners. Please comply with special product information. Only use a soft brush or soft sponge. Examples of plastic parts are the parking brakes, footplates, and ... .

### 4.3.3 Coating

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

When cleaning, only use warm water and normal household detergents and soft brushes and cloths. Ensure that no wetness gets into the tubes.

Initially the zinc parts only require rubbing with a dry cloth. Stubborn dirt can best be removed by using a suitable commercial zinc polish.

## 4.4 Inspection

In principle we recommend one inspection every year, and a minimum of one before usage is resumed. All of the following checks must be performed and documented by authorized persons:

- Check the frame parts and the hinged tubes for plastic deformation, cracks and impaired functioning.
- Visually check for damage to the paintwork (danger of corrosion)
- Check the operation of the wheels (free running, level rolling, axle play, tyres, profile, condition of the rims, air pressure in the case of air tyres, floating axles, etc.)
- Check the solidity and seating of all screws.
- Verify the amount of grease on the metal joints of movable parts
- The condition and security of the guides and axles of the steering wheels
- Visually check all plastic parts for cracks and brittle spots
- Check the functioning of the arm supports and footrests (locking, load, deformation, wear and tear caused by loads, adjustment levers).
- Check the functioning of the gas pressure springs and rope pulls (kinks, foreign bodies), levers and adjusting screws. If necessary, adjust the rope pulls using the adjusting screws.
- Check the functioning of all adjusting mechanisms.
- Check the operation of other detachable parts (example: head support, personal safety belt, accessories, etc.)
- Completeness of the delivery condition, instruction manual available?

The service must only be signed off in the maintenance plan if a minimum of all the above-mentioned aspects have been checked.

## 4.5 Disinfection

**⚠ WARNING: Dangerous Products - The use of disinfectants is restricted to authorized personnel.**

**⚠ WARNING: Dangerous Products, change on irritate your skin - You should wear suitable protective clothing because the disinfectants could irritate your skin. For this purpose you should also take note of the product information of the solutions concerned.**

All parts of the wheelchair can be treated by scrubbing with a disinfectant.

All steps taken to disinfect rehabilitation equipment, their components or other accessory parts are to be recorded in a disinfection report containing a minimum of the following information (with product documentation appended):

Date of the disinfection	Reason	Specification	Substance and concentration	Signature
--------------------------	--------	---------------	-----------------------------	-----------

*Table 4: Example of a disinfection book*

**Abbreviations used in column 2 (reason):**

**V = Suspected infection IF = Infection case W = Repetition I = Inspection**

For a blank sheet of a disinfection book see § 9.

The recommended disinfectants for scrubbing (based on the list provided by the Robert Koch Institute, RKI) are standing in the table below. The current state of the disinfectants included in the RKI list can be obtained from the Robert Koch Institute (RKI) (homepage: [www.rki.de](http://www.rki.de)).

Active substance	Product name	Laundry disinfection		Surface disinfection (scrubbing-/wiping disinfection)		Disinfection of excretions 1 part sputum or stools + 2 parts diluted solution or 1 part urine + 1 part diluted solution						Area of effectiveness	Manufacturer or Supplier
		Diluted solution	Time to take effect	Diluted solution	Time to take effect	Sputum		Stools		Urine			
						%	Hr.	%	Hr.	%	Hr.		
Phenol or phenol derivative	Amocid	1	12	5	6	5	4	5	6	5	2	A	Lysoform
	Gevisol	0,5	12	5	4	5	4	5	6	5	2	A	Schülke & Mayr
	Helipur			6	4	6	4	6	6	6	2	A	B. Braun
	m-cresylic soap solution (DAB 6)	1	12	5	4							A	
	Phenol	1	12	3	2							A	
Chlorine, organic or inorganic substances with active chlorine	Chloramin-T DAB 9	1,5	12	2,5	2	5	4					A <sup>1</sup> B	
	Clorina	1, 5	12	2,5	2	5	4					A <sup>1</sup> B	Lysoform
	Trichlorol	2	12	3	2	6	4					A <sup>1</sup> B	Lysoform
Per combinations	Apesin AP100 <sup>2</sup>			4	4							AB	Tana PROFESSIONAL
	Dismozon pur <sup>2</sup>			4	1							AB	Bode Chemie
	Perform <sup>2</sup>			3	4							AB	Schülke & Mayr
	Wofesteril <sup>2</sup>			2	4							AB	Kesla Pharma
Formaldehyde and/or other aldehydes or derivatives	Aldasan 2000			4	4							AB	Lysoform
	Antifect FD 10			3	4							AB	Schülke & Mayr
	Antiseptica surface disinfection 7			3	6							AB	Antiseptica
	Apesin AP30			5	4							A	Tana PROFESSIONAL
	Bacillocid special			6	4							AB	Bode Chemie
	Buraton 10F			3	4							AB	Schülke & Mayr
	Desomed A 2000			3	6							AB	Desomed
	Hospital disinfectant cleaner			8	6							AB	Dreiturm
	Desomed Perfekt			7	4							AB*	Desomed
	Formaldehyde-solution (DAB 10), (formaline)	1,5	12	3	4							AB	
	Incidin Perfekt	1	12	3	4							AB	Ecolab
	Incidin Plus			8	6							A	Ecolab
	Kohrsolin	2	12	3	4							AB	Bode Chemie
	Lysoform	4	12	5	6							AB	Lysoform
	Lysoformin	3	12	5	6							AB	Lysoform
	Lysoformin 2000			4	6							AB	Lysoform
	Melsept	2	12	4	6							AB	B. Braun
	Melsitt	4	12	10	4							AB	B. Braun
	Minutil	2	12	6	4							AB	Ecolab
	Multidor			3	6							AB	Ecolab
Nüscosept			5	4							AB	Dr. Nüsken Chemie	
Optisept			7	4							AB*	Dr. Schumacher	
Pursept-FD			7	4							AB*	Merz	
Ultrasol F	3	12	5	4							AB	Fresenius Kabi	
Amphoteric surfactants (amfotensiden)	Tensodur 103	2	12									A	MFH Marienfelde
Lye	Lime-milk <sup>3</sup>							20	6			A <sup>3</sup> B	

1 Not effective against myco-bacteria when service disinfecting, especially in the presence of blood.

2 Not suitable for disinfecting blood-contaminated or porous surfaces (e.g. raw wood).

3 Useless for tuberculosis; preparation of Lime-milk: 1 part dissolved lime (calcium hydroxide) + 3 parts water.

\* Checked for effectiveness on viruses in accordance with checking methods of the RKI (Federal Health Reporting 38 (1995) 242).

A: Suitable for killing vegetative bacterial germs including myco-bacteria as well as fungi, including fungal spores.

B: Suitable for deactivating viruses.

Table 5: Disinfectants

Kindly consult your specialist dealer if you have queries on matters related to disinfection; he will gladly assist you.

## 5 Guarantee

Excerpt from the "General Business Conditions":

(...)

5. The guarantee period for warranty claims is 24 months. As a result of our superior quality requirements, we can increase the time bar on warranty claims beyond these fundamental, statutory requirements for

(...)

-Frame and cross-hinge of wheelchairs 3 years

The guarantee excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the guarantee excludes parts or working parts subject to natural wear and tear.

(...)

## 6 Disposal

The manufacturer is responsible for taking back and recycling the wheelchair. You can find out from your local waste disposal company where you can take the wheelchair to be recycled free of charge. It may not be disposed of with domestic waste.

Your dealer will also be able to advise you should you have any queries.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

## 7 Declaration of conformity

**The manufacturer or his authorized representative :**

N.V. VERMEIREN N.V

**Address :**

Vermeirenplein 1/15  
2920 Kalmthout  
Belgium

**declares under his sole responsibility that the CE marked medical devices :**

Productgroup: Manual wheelchairs  
Brand: Vermeiren  
Type: Inovys II

**have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,**

**and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :**

Medical devices directive MDD 93/42/EEC: 2007

**and is in conformity with the relevant European harmonized standards:**

EN 12182: 1999, EN 12183: 2009

**8 Maintenance plan**

Date	Maintenance	Remarks	Paraph
1/1/2011	Greasing and general servicing	non	

**9 Disinfection book**

Date of the disinfection	Reason	Specification	Substance and concentration	Signature

Abbreviations used in column 2 (reason):  
**V = Suspected infection IF = Infection case W = Repetition I = Inspection**

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## E

**CO**  
We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

**APPL**  
In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

**EXCEPT**  
This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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N.V. VERMEIREN N.V.  
Vermeirenplein 1/15  
B-2920 Kalmthout  
BELGIUM

## SERVICE

The wheelchair was serviced:  
La carrozzina è stat ispezionata:

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### Vermeiren Suisse S.A.
### Vermeiren Nederland B.V.